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**Instruction Manual
Nursing Bed
INOVIA II /
INOVIA II 100**

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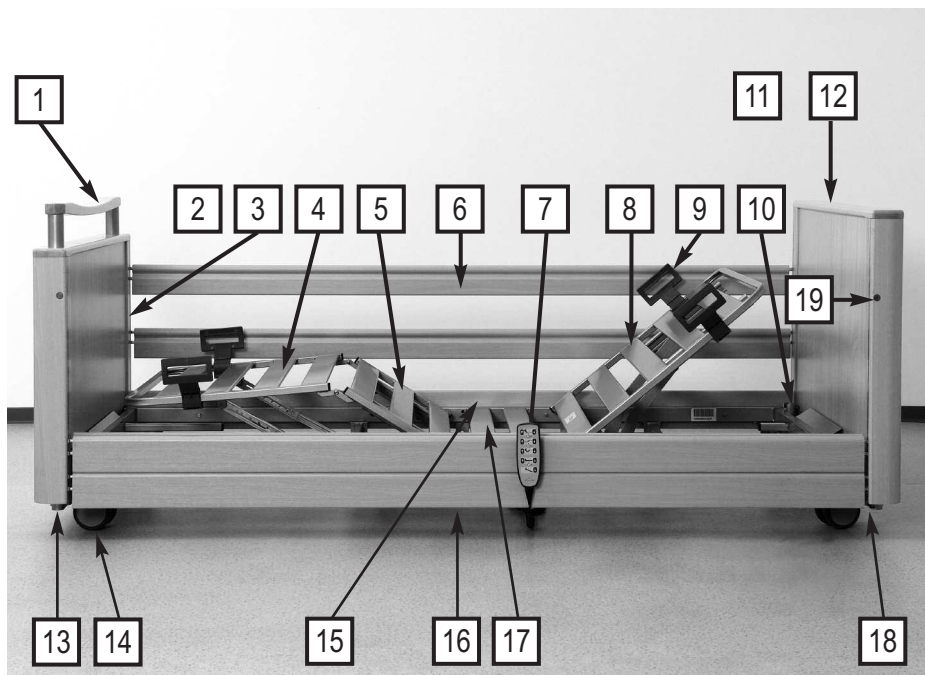
Please notice:

In the text of these instructions square brackets and bold figures = [00] refer to the parts of the nursing bed.

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PREFACE

Dear Customer,

Burmeier has built this bed to give you the best possible help in meeting all the challenges posed by care-giving. We passionately pursue the goal of developing products that are of a high-quality and durable. Our beds should make staying in bed as pleasant and as safe as possible for occupants and also ease the work of care staff and relatives. For this reason, all functions and electrical safety are tested prior to delivery. Each bed leaves our factory in perfect condition.

Correct operation and care are necessary to ensure the bed is kept in this excellent condition over the long term. Please therefore read and observe these operating instructions carefully. They will help you to put the bed into service for the first time and to use it on a daily basis. The instruction manual contains all necessary information to ensure ease of operation and safe handling of this bed, both for you as the operator and for your users. It is a practical reference book and should be kept close to hand at all times.

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Burmeier GmbH & Co.

Kommanditgesellschaft

The medical retail trade that delivered this bed is also there to assist you with any questions you may have concerning servicing and repairs during the product's lifetime of use.

This bed is designed to give the person in need of care and all users a safe and practical piece of equipment that provides decisive support with the ever-increasing requirements of care-giving.

Thank you for the confidence you have place in us and our products.

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www.burmeier.com

You are a medical product retailer and would like to get in touch with Burmeier?

Feel free to phone us: Our customer care centre in Germany can be contacted at +49 (0) 5232 98 41 - 0. Retailers outside Germany can contact our sales distributors in their respective country with any questions they may have. Please visit our website for further information:

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GENERAL NOTES

These instructions may describe functions and accessories which are not included with your model.

Overview INOVIA II Models:

INOVIA II

Model Number: 50.5522.24
Metal Reclining surface, 4 parts
Wooden side rails, 4 bars

INOVIA II 100

Model Number: 50.5522.34
Metal Reclining surface, 4 parts
Wooden side rails, 4 bars

Before putting the nursing bed into operation for the first time:

- Read these instructions carefully in order to avoid damage or incorrect operation.
- Follow the references for authorized mattresses according DIN 13014 (see page 44).
- Clean and disinfect the nursing bed before first use.

According to the "Medizinprodukte-Betreiberverordnung (MPBetreibV) § 2, before using the nursing bed, the user shall ensure that it is in proper working order and free of defects, and be aware of the instruction manual. This applies also to accessories.

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The nursing bed INOVIA II / INOVIA II 100 complies with all requirements of the guidelines 93/42/EWG for medical products. It is classified as a dynamic medical product of the class I in accordance with "Medizinproduktegesetz (German abbreviation: MPG § 13, Medical Products Act)."

The nursing bed INOVIA II / INOVIA II 100 is checked by an independent examining institute. As with all technical, electrical devices improper handling can lead to damage and/or injury.

Observe your obligations as operator in accordance with medical devices "Medizinprodukte-Betreiberverordnung (German abbreviation: MPBetreibV, Operators of Medical Products Ordinance)" in order to ensure a dependable and safe operation of this medical device without endangering patients, users and third parties.

This manual contains safety guidelines, which must be observed. All persons who work with the nursing bed INOVIA II / INOVIA II 100 must familiarize themselves with these instructions and follow the safety guidelines.

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DEFINITIONS OF INVOLVED PERSONS

In this manual the following persons are involved:

Operator

Operators (e.g.: Medical Supply Houses, Dealers and representatives, health insurance companies) are natural persons or legal entities, who utilize the nursing bed INOVIA II / INOVIA II 100 or authorize its use. The operator is responsible for instructing the user in the proper use of the unit.

User

Users are persons who as a result of training and experience are authorized to assemble, perform maintenance and/or operate the nursing bed INOVIA II / INOVIA II 100. Furthermore, the user can recognize and avoid possible dangers, and judge the clinical condition of the patient.

Patient

In this manual a patient is an infirm or handicapped person who reclines on the nursing bed.

Specialist staff

Specialist staff are employees of the operator who are authorized as a result of their education and training to deliver, assemble, dismantle and transport the nursing bed. Furthermore, they are familiar with the regulations for cleaning and disinfecting the unit.

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SAFETY INSTRUCTIONS

The nursing bed INOVIA II / INOVIA II 100 represents the latest technological development at the time of delivery and is checked by an independent examining institute.

■ Do not use the nursing bed INOVIA II / INOVIA II 100 if it is damaged.

MEANING OF THE SAFETY SYMBOLS

In these instructions the following safety symbols are used:

Warning about injuries to persons

Dangerous voltage. Life Threatening.

General danger. Injury or life hazard.

Warning about property damages

Possibility of damage to drive, material or environment.

Other symbols

Useful tip. For easier operation or better understanding of the unit.

The safety symbol used does not replace the text of the safety note. Therefore read the safety instructions and follow them exactly!

SAFETY INSTRUCTIONS FOR OPERATORS

■ Remember your responsibilities in accordance with the MPBetreibV in order to ensure the safe operation of this unit without endangering patients, users and third parties.

■ Before first use, instruct every user in the safe operation of the nursing bed in accordance with these safety instructions, which must be supplied along with the unit.

■ According to MPBetreibV 9 the user shall be notified of the location of this instruction manual.

■ Call attention to the dangers of improper use of the unit, especially in regards to the electrical drives and side rails.

■ The unit should only be operated by properly trained personnel.

■ Make sure that representatives are also familiar with the operation of the nursing bed.

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- During long term use it is recommended that the nursing bed be inspected for function and visible damage at regular intervals (Recommendation: yearly) (see chapter 6).
- Connect the bed directly to an electrical outlet. Do not use extension cords and/or multiple outlet extension cords.
- When attaching other equipment (e.g. compressors for positioning systems) make sure that they are mounted and can function in a secure and safe manner.
- Do not put multiple outlet electrical extensions under the nursing bed. Leaking liquids can cause a fire hazard.
- Pay special attention to:
Securely locating of all wiring, cables, tubes, etc.
- ▶ See page 10 : *Intended use*
- ▶ See page 21 : *Safety note for the electric drive system*
- For further information please contact the manufacturer of the accessories or BURMEIER directly.
- Make sure that your personnel follow the safety instructions.

SAFETY INSTRUCTIONS FOR USERS

- Let the operator instruct you in the safe use of the nursing bed.
- Before each use make sure that the nursing bed is in proper and faultless condition.
- Take care that no obstacles, such as furniture or slanted ceilings interfere with the adjustment functions.
- Pay attention that when using additional electrical components, such as patient lifts, reading lights or compressors for positioning systems that their electrical cords do not get entangled or damaged by the moving parts of the bed.
- Connect the bed directly to an electrical outlet. Do not use extension cords and/or multiple outlet extension cords.
- Make sure that when attaching other appliances (e.g. compressors for positioning systems) that a safe mounting and function is guaranteed.
- Do not put multiple electrical outlets under the nursing bed. Leaking liquids can be a fire hazard.
- Pay special attention to securely locating all wiring, cables, tubes, etc.

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For further information please contact the manufacturer of the accessories or BURMEIER directly.

■ Take the nursing bed out of operation if damage or a malfunction is suspected:

- ▶ Immediately unplug the electrical plug from the wall outlet;
- ▶ Clearly mark the nursing bed as "DEFECTIVE";
- ▶ Promptly notify the appropriate operator.

A checklist for assessing the proper condition is located in Chapter 6.

■ Route the transformer unit cable in such a way that it cannot be stretched, driven over or damaged by moving parts when the nursing bed is operated.

■ Before relocating the nursing bed, it is important that the power cable is removed from the wall socket and the transformer unit is attached to the bed in such a way that the cable cannot fall off or trail along the floor.

■ Check the connecting cable regularly for damage (abrasions, exposed wires, kinks, pressure marks, etc.) in particular:

- ▶ after every larger mechanical strain (e.g.: rolling over the connecting cable with the nurs-

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ing bed or with an equipment cart; after a strong pulling or bending load caused by the bed rolling away with the connecting cable still attached to the wall outlet).

- ▶ after relocating/moving and before plugging in the power cable.

- ▶ during prolonged operating by the user.

■ Do not use the area under the nursing bed as a "parking spot" for any utensils.

■ Lower the reclining surface to its lowest position when leaving the patient unattended. This reduces the risk of injury to the patient getting in and out of the nursing bed.

Please note that the floor clearance of the bed is restricted when the bed is in the lowest horizontal position!

■ Keep the hand switch safe from accidentally falling during non-use (Hang it on the hook). Take care that the cable is not damaged by the moving parts of the nursing bed.

■ To protect the patient and especially children from unintentionally adjusting the hand switch, place it beyond their reach (e.g. at the foot end) or lock the hand switch when:

- ▶ the patient is not able to control the bed safely or is unable to get out of dangerous positions without help;
- ▶ the patient could be endangered, because of unintentional adjustments to the drive motor;

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- ▶ the side rails are up: (There is a possible danger that limbs could be squeezed when adjusting the back- and thigh rest);
- ▶ unsupervised children are in the room with the nursing bed.
- Adjustment functions can only be performed by a trained person or in the presence of a trained person.
- The failure of a lifting motor can result in the bed moving into the Trendelenburg position (head end lowered), which could endanger the patient. If this occurs, the bed must be returned to a horizontal position. Lock the lifting function and inform the operator immediately.

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PRODUCT DESCRIPTION

NORMAL OPERATING PROCEDURE

■ The nursing bed INOVIA II / INOVIA II 100, hereafter called bed, serves as a comfortable solution for positioning and easing the care of infirm, frail people in senior or nursing homes. Additionally it was developed as a comfortable solution for in-home care of infirm, handicapped or frail people. It will support this care.

■ The INOVIA II / INOVIA II 100 care bed is designed for use in hospitals only in rooms of application group 0.

■ This bed can be used for care under the instructions of a physician and can serve in diagnosing, treatment and observation of a patient. It is therefore equipped with a locking mechanism for the electrical adjustment functions.

■ This bed has no special connector possibilities for a potential equalization. Consider this before combining with additional electrically operated (medical) devices.

Further references about additionally applicable precautions can be found:

- ▶ In the instructions for the additional electrically operated devices (e.g. compressors for positioning systems, infusion pumps, nutrition probe ...)
- ▶ In the Standard DIN EN 60601-1-1:2002 (Safety of medical electrical systems)
- ▶ In the Standard VDE 0107:1994 (High current systems in hospitals)

■ The maximum long-term load for this bed is 225 kg (patient and accessories). Under this load, the bed can be operated without restriction.

■ The permitted weight of the patient depends on the total weight of the accessories attached at the same time (e.g. respirators, infusions,...)

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Example:

Weight of accessories (with mattress):	Max. permitted weight of patient
10 kg	215 kg
40 kg	185 kg

▶ Observe, especially with patients in poor health condition, the safety tips in chapter 4.

■ This bed may only be operated by trained personnel.

■ This bed is useful for multiple re-use. Bear in mind the necessary requirements:

- ▶ See chapter 5;
- ▶ See chapter 6.

■ While the patient also is lying in bed, it is allowed to drive the bed within the building.

This bed may only be operated under the operating instructions laid out in this instruction manual. This bed must not be altered in any way. Any other use is deemed to be inappropriate.

Sale of this product in North America, especially in the United States of America, is prohibited. Distribution and use of the nursing bed, including distribution and use by third parties in these markets is forbidden by the Manufacturer.

CONTRAINDICATIONS

■ In case of using safety sides this bed is only suitable for patients who do not fall below the following minimum body size/weight:

- ▶ Height: 146 cm
- ▶ Weight: 40 kg
- ▶ Body Mass Index „BMI“: 17

Calculation of BMI =

Weight of patient (kg) / Height of patient (m)²

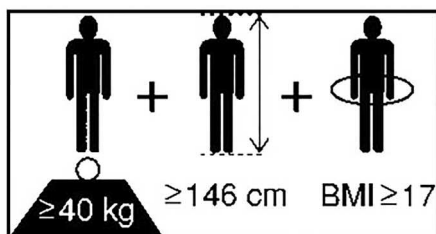
Example a

41 kg / (1.5 m x 1.5 m) = 18.2 = **OK!**

Example b

35 kg / (1.5 m x 1.5 m) = 15.6 = **Not ok!**

■ Owing to the smaller limbs of patients with lower body measurements/weight, there is an increased risk of entrapment between the open spaces of the safety sides when safety side systems are used.

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STRUCTURAL DESIGN

The bed is shipped disassembled. It can be transported even in apartment homes. It consists of two end boards (head and foot boards) which are each made up of a tubular frame encased with wooden boards; an encased reclining surface frame and four side rail beams. The bed rests on steerable castors with individual axle braking.

Reclining surface frame

The reclining surface is divided into four sections: a removable back rest, a fixed middle section and a movable thigh rest and lower leg rest. The back- and thigh rest can be adjusted with the aid of electric motors. The horizontal height of the reclining surface can be adjusted and inclined to a low foot end position. All adjustment functions are controlled by a hand switch.

Side Rails

The bed can be fitted with a variety of different safety sides.

Electric adjustment system

The electrical adjustment system of this bed is error protected, flame-retardant (V0) and consists of:

- the central drive unit with the drive motors for the back and thigh rest. An external transformer creates a low voltage, which is safe for patient and user. All drive motors and the hand switch are hooked up to the central drive unit by plug-in connections and work with the safety low-voltage;
- two drive motors for the height adjustment of the reclining surface;
- a lockable hand switch with a strong hook.

DESCRIPTION OF MATERIALS

The bed is mostly constructed out of steel sections, their surfaces have been polyester-powder-coated, or with a metallic coating of zinc and chrome. The end boards, the side rail beams and the spring wood slats consist of wood and derived timber products whose surface has been sealed.

All surfaces are harmless to skin contact.

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Beipack: Montagematerial

ASSEMBLY AND START UP PROCEDURE

The assembly is taken care of at the destination by qualified personnel supplied by the operator.

The bed ships in 5 cartons:

Carton 1: Reclining surface frame including drive motors (pre-assembled)

Carton 2: Tubular frame head section with drive motor

Carton 3: Tubular frame foot section with drive motor

Carton 4: 4 side rail beams; 2 side boards

Carton 5: Wooden boards, head and foot section

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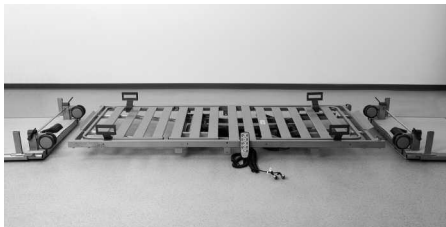
ASSEMBLY

PREPARATION

☞ Position the reclining surface frame [17] flat on the floor. Top side up.

☞ Position the tubular head section frames [2] + [11] flat on the floor. The castors point upward.

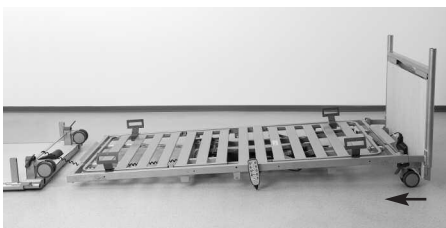
☞ Connect both drive motors of the end boards to the drive unit. (Electrical connection see page 16).

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
ASSEMBLY OF THE RECLINING SURFACE FRAME

Start the assembly at the head section. Join the tubular frame/head section [11] with the reclining surface frame [17] together.

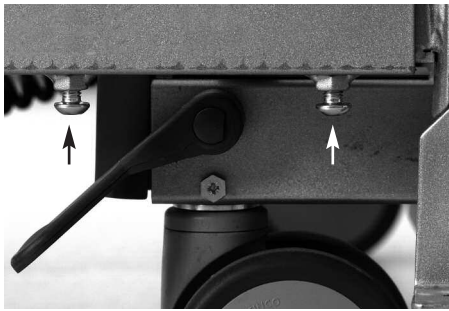☞ Lift the reclining surface frame at the head end and push the two support brackets from the tubular frame/head section [11] into the tubes of the reclining surface frame [17] to the limit.

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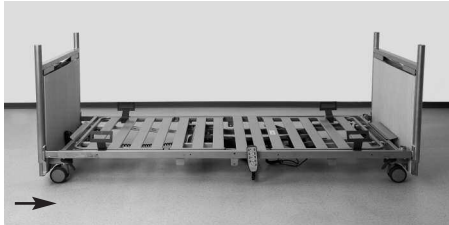
☞ Screw the reclining surface frame firmly to the support brackets.

For this use two Allen screws M 8 x 20 mm on each side.

☞ Tighten the 4 Allen screws firmly with a size 5 Allen wrench.

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☞ Repeat the process with the tubular frame/foot section [2].

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ASSEMBLY OF THE SIDE BOARDS

☞ Raise the reclining surface to the middle horizontal position (see page 22).

☞ Raise the back and thigh rests [8] + [5] half way. Lift the lower leg rest [5] to the horizontal position (see page 24).

Notice: The two side boards are identical.

The castors must be locked.

☞ Screw the side boards [15] to both sides of the reclining surface frame with 4 Phillips head screws respectively; fit the middlest screw first.

Take care, that the low side of the side board point upward.

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ASSEMBLING THE SIDE RAIL GUIDE

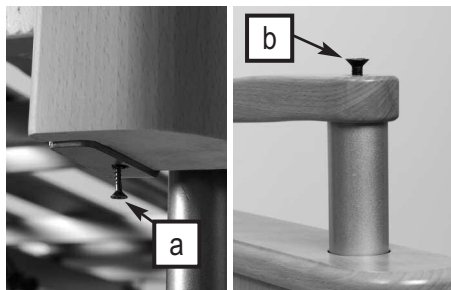
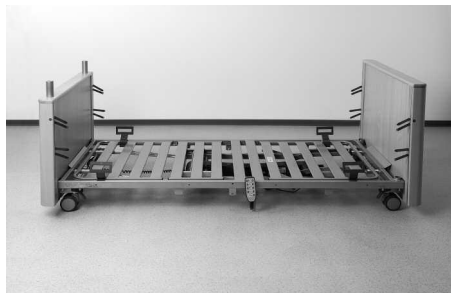
On each corner of the wooden boards [5] + [14] you will find a guide bar embedded in the wooden boards for the side rail beams [6].

When inserting the side rail guide, please adhere to the following procedure:

☞ Insert the plastic slide from below into the guide bar. The pointed end of the plastic slide must face upward.

☞ Insert the sliding plastic retainers for the safety sides right into the guide bars on the head and footboards.

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ASSEMBLY OF THE WOODEN BOARDS

☞ Finally put the wooden board/head section [12] and the wooden board/foot section [3] respectively, on the tubular frame head and foot section [11] + [2].

Insert the wooden boards all the way.

☞ Tightly screw the lower section of the two wooden boards [3] + [12] on both sides to the tubular frame [2] + [11] with the M 6 x 30 mm [a] Philips head screws.

☞ Screw on the strip handle on the foot board using screw [b].

These steps of procedure have to be made only for the initial assembly.

For further assemblies/disassemblies the wooden boards should stay on the tubular frame.

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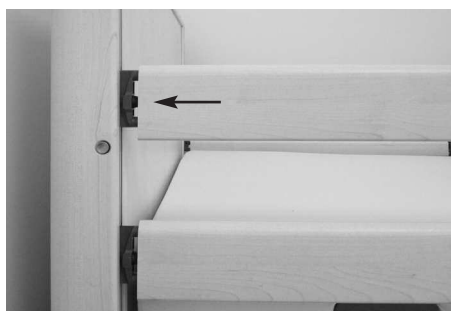
ASSEMBLY OF THE SIDE RAIL BEAMS

On all four corners of the bed you will find a guide bar embedded in the wooden boards [3] + [12] for the side rail beams [6].

In each of these runs a plastic slide with 4 plastic plugs. The plastic plugs fit into the slots on the ends of the 4 side rail beams.

The side rail beams can be raised or lowered, as needed.

Start with the end board/head section:

☞ Fit the side rail beams onto the plastic plugs.

The top and bottom of the side rail beams are flattened and rounded respectively. The round side of the bars must point upward.

Now go to the end board/foot section.

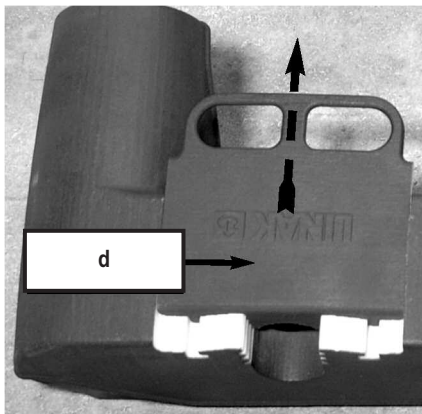
☞ Loosen the Allen screws on the reclining surface frame.

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- ☞ Pull the end board and the reclining surface frame apart.
- ☞ Fit the side rail beams onto the plastic plugs.
- ☞ Mount the two other side rail beams on the opposite side of the bed the same way.
- ☞ Push the end board and the reclining surface frame together.
- ☞ Tighten the 4 Allen screws firmly with a size 5 Allen wrench.
- ☞ Screw the reclining surface frame to the support plates.
- ☞ Push the side rail beams upwards until they click into place.

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

ASSEMBLY OF THE DRIVE UNIT (PRE-ASSEMBLED)

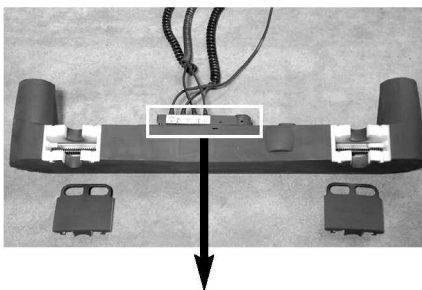- ☞ Unpack the drive unit [16] and all connecting cables.
- ☞ Place the drive unit on the floor with the two cover slides [d] uppermost.
- ☞ Using the finger holes, pull the two cover slides [d] off laterally and lay them aside.

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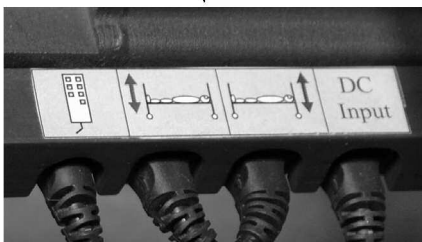- ☞ Insert the 4 plugs of the respective cables, 2 spiral, the 24-V-connector cable and the hand switch cable [7] into the marked sockets of the drive unit [16].

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The plugs are polarised.

The two spiral cables for the drive motors are identical.

Push the plugs firmly into the sockets to seal against moisture.

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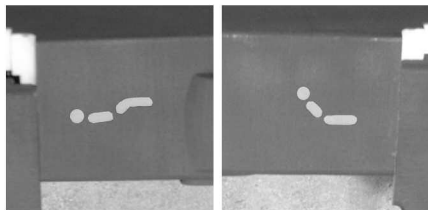
☞ Place the reclining surface frame upright with head part down in order to work on the underside.

Warning! Secure the frame in this position.

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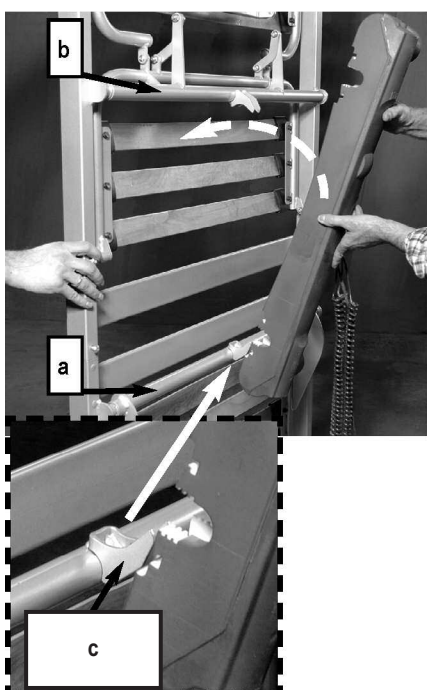
☞ Retrieve the two cover slides and to pocket them.

☞ Take the drive unit [16] in both hands.

The drive unit has two symbols on the housing for correct orientation.

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☞ Find the "backrest" symbol and set the drive unit with its attachment hole over the metal claws on the crossbar [a] of the backrest.

The metal claws should snap easily but securely into the drive unit.

☞ Swing the drive unit up until the other attachment hole (with the "thigh rest" symbol) receives the metal claws [c] on the crossbar [b] of the thigh rest.

The metal claws should snap easily but securely into place.

☞ Hold the drive unit with one hand and push the two cover slides back into place with the other.

☞ Place the reclining surface frame assembly on the floor to one side with the drive unit facing up.

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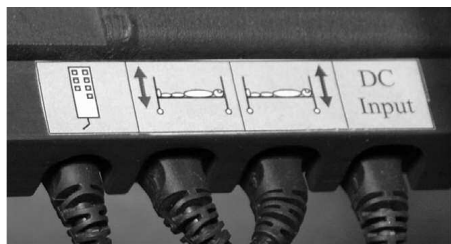
ELECTRICAL CONNECTION

Take special care in assigning the drive motors for the height adjustments. These must not be interchanged. Only one lower foot position should be possible.

☞ Place the drive motor cables and the hand switch cable under the reclining surface frame.

☞ Hang the drive unit cable in the corresponding holder on the foot and head boards.

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The connecting cable for the electric motors must be routed when the nursing bed is at the highest horizontal position to avoid straining the cable.

Make sure that no cable is damaged, that there is no loop in the cables and that the cables are not crushed by moving parts.

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When exchanging single electrical components, always make sure that their plugs have been plugged back into the central drive unit until they lock and the plug covering strips have been re-affixed.

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Special care is to be taken when positioning the connecting cable. Make sure that the cable is not damaged and there is no loop in it! When moving the bed around, take care that the castors do not run over the connecting cable! When adjusting the bed the power cable should not get caught between moving parts!

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LOCATION REQUIREMENTS

■ There must be enough space available to be able to adjust the bed. No furniture, window sills and the like should be in the way.

■ The space under the bed must remain clear.

■ Check before use on parquet flooring, that the castors do not cause discoloration of the varnish. Use on ceramic tiles, carpet, laminate and linoleum is harmless.

BURMEIER is not liable for any damage to floors due to daily operation.

■ A properly installed 230 volt power outlet must be available, near the bed.

■ Take care when attaching further accessory equipment (e.g. compressors for positioning systems, etc.) that their attachment is safe and their functionality is not impaired. Especially important is hereby also the safe placement of all movable connecting cables, tubes, etc. If you have any further questions or need clarification contact the manufacturer of the accessory equipment or BURMEIER.

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Observe the following points when setting up the bed to minimize possible fire hazard.

Instruct the user and the authorized personnel in these points!

- If possible use only fire-retardant mattresses and bed sheets.
- Only use appropriate firm mattresses according to DIN 13014. This mattress should be inflammable according to DIN 597 Section 1 and 2.
- Avoid smoking in bed because of the danger of fire.
- Use only technically faultless appliances (e.g. heating blankets) and other electronic devices (e.g. lamps, radios).
- Take care that these devices are only used for the purpose intended and that they cannot accidentally get on top of or underneath the bed sheets (Fire hazard because of overheating).
- Absolutely avoid the use of extension cords or multi outlet extension cords under the bed (Leaking liquids can cause a fire hazard).
- Connect the bed directly to an electrical outlet. Do not use extension cords and/or multiple outlet extension cords.

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REACHING OPERATIONAL READINESS

- After successful assembly, complete an inspection according to chapter 6.
- Clean and disinfect the bed before first time use and before every further use according to chapter 5.

After successfully completing and heeding all steps on page 12 through 19 the bed is now operationally ready.

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DISASSEMBLY

- ☞ Lower the bed into the middle horizontal position.
- ☞ Pull the power plug out of the socket.
- ☞ Remove the security screws in the guide bars.
- ☞ Remove all side rail beams consecutively.
- ☞ Insert the plastic slide back into the guide bar.

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☞ Loosen the screws and raise both wooden boards from the tubular frames of end boards one by one.

☞ Loosen the Allen screws from the reclining surface frame on the foot section, to remove the tubular frame from the foot section; next loosen the Allen screws from the reclining surface frame on the head section, to remove the tubular frame from the head section.

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OPERATIONS

GENERAL SAFETY INSTRUCTIONS FOR OPERATION

■ In order to avoid toe injuries, wear closed shoes (and not sandals) when operating the bed.

■ Make sure that the castors have their brakes applied.

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SPECIAL SAFETY INSTRUCTIONS FOR THE DRIVE UNIT

■ Take care when adjusting the bed that no limbs of patients, users or any other persons, especially children, can be jammed or injured by getting under the movable parts or the bed frame.

■ For the safety of the patient, lock the hand switch operation of adjustments to the back and thigh rests to avoid unwanted adjustments when the side rail beams are up. (Danger of getting limbs trapped when adjusting the back- and thigh rests).

■ When using accessories with the bed it is necessary to: make sure that the arrangement of these accessories will not cause any places that might squeeze or hurt the patient. If this is not the case the operator must safely disable the adjustment function of the back- and thigh rests.

► To do so place the hand switch out of the patient's reach (e.g. foot section) or lock the adjustment functions on the hand switch.

■ Make sure that the power and hand switch cable cannot be jammed or damaged in other ways.

■ Ensure that only hand switches that are in perfect condition are used. Faulty hand switches (e.g. with torn or indented foil) can result in spontaneous operation of the bed and can thus endanger the patient.

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■ Make sure that before relocating the bed, the connecting cable cannot be stretched, rolled over or damaged in any other way. Before relocating the bed, ensure that the connecting cables are securely positioned on the bed so that it cannot fall off.

■ Make sure that when adjusting the bed no obstacles such as furniture or slanted ceilings are in the way. This will prevent damage.

■ Each time you wish to transport the bed, make sure that you unplug the transformer unit first and place it on the bed in such a way that the cables cannot hang down and touch the floor. Lay them safely on the mattress base (or mattress).

Failure to do this could cause permanent damage to the 230 volt connection plug.

■ To prevent the ingress of moisture, always fit the lid to the plug coupling when the transformer unit is not connected to the bed.

■ Ensure that the connecting cable and the hand switch cable cannot be driven over or crushed when moving the bed.

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♦ An electronic overload detector switches off the central control unit, when the load is too great. After clearing the overload and pressing the hand switch again the central control unit will work again.

♦ A continuous run time of 2 minutes should not be exceeded. Afterwards a pause of no less than 18 minutes should be observed. Another possibility: 1 minute continuous run time and 9 minutes pause.

♦ If the maximum continuous run time is not heeded (e.g. by constantly "playing" with the hand switch) a thermal safety component switches off the power for safety reasons. Depending on how large the over-load was, it can take several minutes before any more adjustments can be made.

♦ The adjustable range of every function is electrically/mechanically possible only within a predetermined range.

♦ Electrical disturbances affecting other nearby electronic appliances (e.g. crackling on the radio) cannot be excluded, despite adherence to all prescribed regulations. In such rare circumstances increase the distance between the appliances, do not use the same power socket or switch the affected appliance off for a while.

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HAND SWITCH

The electrical adjustments can be made using the hand switch [8].

The adjustable range of every function is electrically/mechanically possible only within a predetermined range.

Locking mechanisms have been built into the hand switch for safety reasons. For the safety of the patient, adjustment functions can be locked at the hand switch, if, from a clinical point of view and with the approval of the attending physician, the condition of the patient warrants it.

- ◆ The hand switch, with its elastic hook can be hooked onto the bed.
- ◆ The spiral cable allows for greater mobility.
- ◆ The hand switch is water-tight and washable (IP 54).

OPERATING THE HAND SWITCH

- Train the patient in the functions of the hand switch.
- The drive motors will run as long as the button is held down.
- The adjustment functions, except for the lowered foot position, can be used in both directions.

- Basically the functions of the buttons are as shown :

- ▲ raise
- ▼ lower

Adjusting the backrest

Push this button if you want to adjust the backrest angle.

Adjusting the thigh rest

Push this button to adjust the thigh rest angle.

Setting up the Auto-contour

Press this button if you want to adjust the angles of the back and thigh rest at the same time. This prevents the patient sliding to the foot end.

Adjusting the reclining surface height

Push this button if you want to adjust the height of the reclining surface.

If the bed is in the lowered foot position, it will automatically be adjusted to horizontal level, when it is readjusted to the lowest or highest position.

Adjusting to the lowered foot position

Push this button if you want to adjust to the lowered foot position. With this button ▼ only the function "Down" is possible.

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If the bed is in the lowered foot position, it will automatically be adjusted to horizontal level, when it is readjusted to the lowest or highest position.

■ Unlock the foot brake of both castors on the head or foot section to avoid damaging the floor.

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■ When the bed is reclined a pinch point can occur at the foot end. For this reason the bed must only be lowered after it has been ensured that there is no person situated in the hazard area.

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LOCKING FUNCTION

Only the operator is authorized to use the locking function!

In case the clinical condition of the patient is so critical that any adjustment of the beds by means of hand switch causes danger, the operator has to disable the bed immediately. The nursing bed remains in the position it had at the time of disconnecting.

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Do not turn the locking key with force over the stop of the lock! The lock or the whole hand switch can be damaged.

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The locking key is delivered attached to the bed with a cable tie on the hand switch. The locking key is not intended for use by the patient. It must be removed from the hand switch. The operator or a person authorized by the physician should keep the **locking key**.

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■ Turn the respective lock on the hand switch with the locking key clockwise in the locking position.

The color of the respective display will change from green to yellow.

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Lock is vertical

Color of the display is: **green**

Buttons can be pushed ("Clicking sound")

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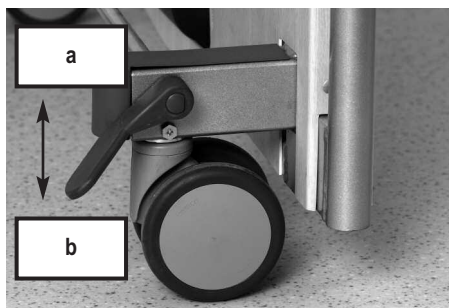
Drive disabled

Lock has been turned clockwise by about 90°

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CASTORS

The bed stands on four steerable, castors [14]. They can be locked in pairs with two foot levers [17].

☞ **Braking [b]:** Push the foot lever down.

☞ **Driving [a]:** Push the foot lever down in the opposite direction. To safely steer it, push the bed only at the foot end.

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■ Make sure that at least all castors have their brakes applied.

■ Relocate the bed only when it is in the lowest position.

■ Before relocating the bed, ensure that the connecting cable and the transformer unit are securely positioned on the bed so that they cannot fall off.

■ The bed cannot be braked when in the lowest horizontal position.

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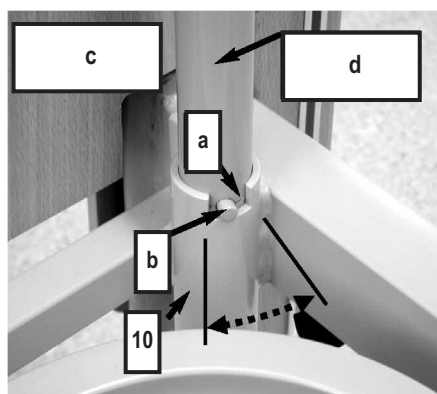
TRAPEZE POLE (ACCESSORIES)

The trapeze pole [d] attached to the bed makes it easier for the patient to get into and out of bed.

■ The maximum load of the trapeze pole is 75 kg at the front end.

Do not use the trapeze pole for rehabilitation training.

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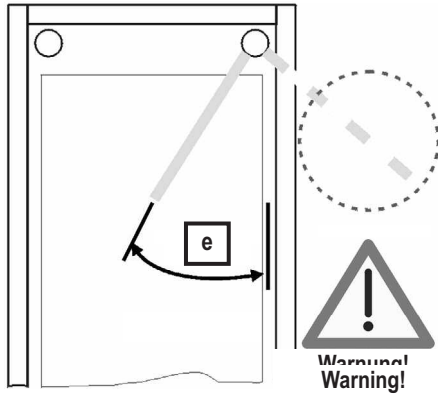
In both corners of the head section [c] of the reclining surface frame are two trapeze pole sockets [10]. The trapeze pole socket has a notch at the top [a], which together with the metal pin [b] restricts the swivel range of the trapeze pole. The trapeze pole should be installed on the side where the patient gets in and out of bed.

Attaching

☞ Insert the trapeze pole into the socket. The metal pin [b] must be in the notch [a] of the socket.

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Removal

☞ Pull the trapeze pole straight up out of the socket.

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Trapeze pole swivel range [e]

Do not swivel the trapeze pole outside of the bed. There is a danger that through the pull, the bed will tip over.

The metal pin must always be in the notch of the socket.

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Durability

There is a date clock on the trapeze. With normal use the trapeze will last for at least five years. Afterwards a sight check is to be performed periodically to determine if further use is permissible.

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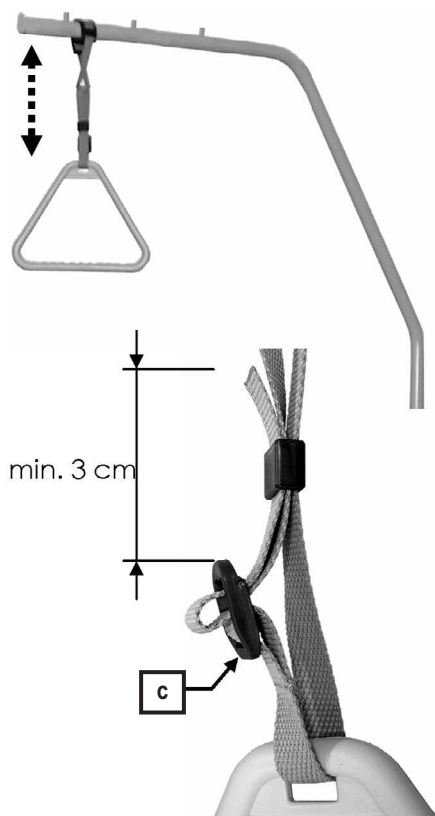
Adjustable range of the trapeze

The height of the trapeze can be adjusted by means of the strap to accommodate a range of about 55 cm to 70 cm (measuring from the top of the mattress).

☞ Push the loop of the trapeze [1] over the first stud of the patient grab handle pole.

☞ Test by tightly pulling the trapeze downward that it is secure.

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Useful Tip: The maximum lifting load of the trapeze pole is 75 kg at the front end.

■ The height of the trapeze can be adjusted by means of the strap.

■ Make sure the strap is fed correctly through the buckle [c].

■ Make sure that the end of the strap is at least 3 cm above the buckle [c].

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LOWER LEG REST

When the thigh rest is fixed, the lower leg rest [5] can be individually adjusted. For this reason there are two adjusting ratchets, which will lock the lower leg rest into position.

Raising

☞ Raise the lower leg rest on the foot end - not with the mattress brackets - up to the desired level. The lower leg rest will automatically snap into place.

Lowering

☞ First lift the lower leg rest all the way to the stopping point.
☞ Slowly lower the lower leg rest.

There is a potential for injury if the lower leg rest falls straight down.

If you lower the thigh rest, the lower leg rest will be lowered automatically.

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SIDE RAILS

Side rails conveniently prevent the patient from falling out of bed accidentally. They do not, however, prevent a patient from leaving the bed.

Engaging

☞ Pull the side rail beams [6] on one end after the other until they lock in its highest position. You should not be able to move them up or down.

☞ Check that the side rail beams are locked into place by pushing on them from above.

Disengaging

☞ Lift the side rail beams slightly.

☞ Press the locking button [19] and slowly lower the side rail beams.

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Special safety notes for use of side rail beams:

■ Only use safely lockable, technically perfect, undamaged side rail beams with proper gap dimensions

■ Make certain that only suitable Original BURMEIER side rail beams are used, which have been approved for use with the bed model you are using or that are already integrated in the bed.

■ Before using side rail beams evaluate and take into account the clinical condition and any special problems of the patient

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► If, for example, the patient is very bewildered or very anxious, forgo the side rails and use alternative safety measures such as a restraining belt or restraint sheets.

► For especially small, frail patients, additional protective measures for reducing the distance between bars on the side rails may be necessary. In these cases use, for example side rail foam covers, retaining belts, etc.

This will effectively ensure patient safety by reducing the risk of becoming trapped or slipping through.

■ Only use authorized mattresses (see page 44). The mattress should be fire-retardant according to DIN 597 Section 1 and 2.

■ If elevated patient positioning systems are employed, such as antidecubitus mattresses, an effective side rail height of at least 22 cm above the unweighted mattresses must be ensured. If this requirement is not met, the operator must take suitable measures to safeguard the patient. This can be done by using an attachable bar for side rails (Accessory).

■ The mechanical parts on the bed frame and the side rails which serve to protect the patient must be regularly examined for damage. This should not only happen before the installation of the side rail but also during use in shorter intervals (at least before every new assignment).

■ To safeguard the patient against unintentional adjustments, always lock the electrical adjustment of the back and thigh rests on electrical beds when the side rails are raised. Otherwise there is a risk of trapping limbs when the back and thigh rest are adjusted.

■ Before lowering the side rail beams make sure that the patient's limbs are clear of the side rail beams. This way you can avoid injuring the patient.

If these safety warnings are not observed there is a danger for the patient of being injured by the side rails, of falling out of bed because of too large a distance between the bars or because of damage, the use of unsuitable side rails or the incomplete locking of the side rail.

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EMERGENCY LOWERING OF THE BACK REST

The upright back rest can be lowered manually in case of a power outage or failure of the power supply or the electrical drive systems.

In this case two users are absolutely necessary!

Disregarding these safety guidelines and instructions may cause the backrest to fall out of control, which could lead to serious injuries for the user and the patient!

Emergency lowering may only be carried out in the case of extreme emergency and by users who have a complete command of the procedures below.

We strongly advise you to practice emergency lowering of the backrest under normal conditions so that in case of an emergency you will be able to react quickly and properly.

☞ Before the emergency lowering, ease the load on the backrest.

☞ The first user raised the back rest slightly by the outside edge of the head section and holds it in this position.

☞ The second user now removes the cover slide on the side of the drive unit that drives the back rest.

☞ Swing the drive unit down and away.

The drive shaft is now separated from the back rest.

☞ Now the first user carefully lowers the back rest.

If the back rest falls out of control, the patient and the second user may be injured!

☞ To return the bed to its original setup:

Swing up the drive unit and let the drive shaft click into place, now replace the cover slide.

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CLEANING AND DISINFECTING

GENERAL

Cleaning is the most important requirement for a successful chemical disinfection.

Generally a routine cleaning of the bed, if used by the same patient, is hygienically sufficient.

Disinfection of the bed frame is only necessary in case of visible contamination with infectious or potentially infectious material (blood, stool, pus) or by presence of an infectious disease under the direction of a physician.

The bed must be cleaned and washed down with disinfectant in case of a patient change!

Before beginning the cleaning, please note:

- Never clean the transformer unit when cleaning the bed. Remove the transformer unit from the socket and avoid contact with water. These components may be cleaned only with a slightly damp cloth.
- If the bed is not connected to a transformer unit, seal the plug coupling on the bed using the attached lid. This prevents faults that could otherwise be caused by moisture.
- Make sure that all the other plugs for the drive components are properly plugged in and secured against being pulled out.
- None of the electrical components must show any visible damage; otherwise water or cleaning solution could penetrate. This could lead to malfunction or damage to the electrical components.
- Before operating the bed again, make sure that there is no residual moisture on the electrical contacts, by drying or blowing on the power plug.
- The electrical components must not be cleaned with a water jet, a high pressure cleaner or other similar device!
Clean only with a moist towel.
- If you suspect that water or any other form of moisture has penetrated the electrical components, pull the power plug out of the wall socket immediately. If already disconnected from the power supply, make sure it is not plugged in again. Clearly mark the nursing bed as "Defective" and take it immediately out of service. Report this to the operator promptly.

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■ Failure to follow these directions could lead to major damage of the bed and its electrical components and lead to subsequent malfunctions.

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CLEANING AND DISINFECTION PLAN

■ Remove the bed sheets and give them to the laundry.

■ Clean all surfaces including the slatted boards and the reclining surface made of plastic inserts or wire mesh base with a mild and environmentally safe cleaning solution. This also applies to the hand switch.

■ If the bed has visible contamination e.g. infectious or potentially infectious material has polluted it, it must be disinfected by wiping after cleaning. Use approved disinfectants for each surface according to the products listed by the DGHM (German Organization for Hygiene and Microbiology). This also applies to all beds from patients with illnesses subject to report according to §6 of the infection control law (German abbreviation: IfSG, Protection against Infection Act), colonization or infection with treatment resistant pathogens (e.g. MRSA, VRE) and all beds from the intensive care and infection control units. Use only the DGHM approved disinfectant in the approved concentration.

■ Disinfecting the castors is only necessary when visibly contaminated by infectious or potentially infectious material.

Notice: Continuous disinfection is only necessary for patients with a treatment resistant pathogen (e.g. MRSA) within the hospital.

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INSTRUCTING THE USER AND QUALIFIED PERSONNEL

To guarantee the correct procedure for cleaning and disinfection is followed, we recommend to correctly instruct the handler and qualified personnel. Thereby communicating that the following points need to be observed:

■ The clean bed is to be transported to the patient's home in such a way that it does not become polluted or contaminated.

■ When disassembling the bed we recommend cleaning and disinfecting it right away. The qualified personnel should be familiar with the steps necessary to thoroughly clean and disinfect (the operator will provide the individual steps necessary). It is necessary to use only the DGHM approved disinfectant in the approved concentration. The disinfectant must be appropriate for the surface being cleaned.

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■ The qualified personnel must be equipped with water-tight one-time-use aprons and gloves.

■ Only fresh and clean towels must be used to clean and disinfect, these must then be given to the laundry.

■ After cleaning the bed, the qualified personnel must also disinfect their hands before going to the next patient. An appropriate disinfectant dispenser should be part of the qualified personnel's gear.

The immediate cleaning of the bed while disassembling has the benefit of not transporting "dirty" beds or bed parts along with clean ones in the same truck. A transfer of potentially infectious spores which may be clinging to the used bed may be avoided. A transfer of spores of a hospital-acquired infection will be avoided with strict adherence to these recommendations.

If the beds are to be stored for a while, they should be covered to keep them free of dust, accidental pollution and contamination.

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CLEANING SUPPLIES AND DISINFECTANTS

Make use of these recommendations to keep the bed serviceable for a long time.

■ The surfaces must be intact; any damage should be repaired immediately.

■ We recommend a (damp) wash down. When choosing the cleaning solution make sure it is mild (gentle to skin and surfaces) and environmentally friendly. In general a regular household cleaner can be used.

■ For cleaning the coated metal parts use a wet cloth with regular household cleaners.

■ The disinfectant used for the wash down should be one approved for the purpose and listed by the DGHM (in the recommended dosage). Disinfectants and concentration listed by the Robert-Koch-Institute (the former Bundesgesundheitsamtes) should only be used in case of an epidemic according to the infection safety law and only on the orders of the appropriate health department.

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■ Despite the excellent mechanical resistance of the bed, scratches, markings, etc. which penetrate the coating should be resealed using a suitable medium to prevent the penetration of moisture. Contact BURMEIER or your specialized dealer with any questions.

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■ Do not use scouring agents, stainless steel cleaners, abrasive cleaning agents or scouring pads. These substances can damage the surface.

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Notice:

Surface disinfectants based on aldehyde generally have the advantage of working on a large array of pathogens, have a relatively low protein error and are environmentally friendly. Its main disadvantage is the allergy-causing and irritation potential. Compounds on the basis of glucoprotamine do not have this disadvantage and are very effective, though mostly more expensive. Disinfectants on the basis of chlorine splitting combinations, with prolonged contact or in too high doses, can be corrosive to metals, plastics, rubber and other materials. In addition to this, the compounds have a high protein error, are irritants to the mucous membranes and are not environmentally friendly.

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CONTACT WITH DISINFECTANTS

■ Follow the guidelines for each product and manufacturer. Observe the exact dosage! We recommend automatic dosage apparatus.

■ Always mix the solution with cold water, to avoid vapors that aggravate the mucous membranes.

■ Wear gloves to avoid direct skin contact.

■ Do not store the mixed disinfectant in open containers with the cleaning rag inside. Close the container!

■ Use closeable bottles with dosage pumps for wetting the cleaning rag.

■ After finishing the surface disinfection ventilate the room.

■ Use a wipe down disinfecting method, do not use a spray. Using the disinfectant as a spray causes the disinfectant to turn into a mist which could be inhaled.

■ The wipe-down disinfection plays a major part.

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operators of nursing beds are obligated to keep medical products in a safe state throughout their entire service life. This also includes regularly carrying out expert maintenance as well as safety checks.

If a bed is purchased for domestic use (not for commercial use), the regular safety inspection is not compulsory but is still recommended by the manufacturer.

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Notice for Operators

This bed has been constructed and built to last safely for a long time. With appropriate operation and use, this bed has a life expectancy of 2 to 8 years. The life expectancy is based on the conditions and frequency of use.

Through repeated transport, assembly and disassembly, improper operation as well as long time use, it is possible that damage, defects and deterioration can occur. These deficiencies can lead to danger if they are not recognized and quickly fixed.

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Because of this there are legal bases for the regular enforcement of tests to ensure the safe condition of this medical product.

According to § 4 of the Medizinprodukte-Betreiberverordnung operators are obliged to maintain the product. Because of this the following regular inspection and function controls are to be performed by the operator as well as the user.

Point out these necessary tests to the user.

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BY THE USER

Along with the regular substantial tests by technically qualified personnel, the normal user (nurses, supporting relatives, etc.) must also perform a sight and function check at shorter, regular intervals, as well as before a new patient.

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■ If damage or a malfunction is suspected, take the bed out of service immediately and disconnect it from the power until an exchange or repair can be made!

■ Contact your operator regarding exchange or repair of faulty parts.

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Recommendation:

All electrical and mechanical components should be tested monthly. The connecting cable as well as the hand switch cable should be checked after every mechanical load and after every relocation.

For this purpose use the checklist on the following **page 36**.

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Checklist: Test protocol by the user

Test		OK	Not OK	Defect description
Visual check of electrical components				
Hand switch and cable	Damage, Cable Routing			
Connecting cable	Damage, Cable Routing			
Hand switch	Damage, Foil			
Visual check of mechanical components				
Trapeze pole/Socket	Damage, Deformation			
Bed frame	Damage, Deformation			
Bed box	Damage, Splinters			
Reclining surface frame	Damage, Deformation			
Side rail	Damage, Splinters			
Function test of electrical components				
Hand switch	Functiontest, Blocking function			
Function test of mechanical components				
Emergency lowering of backrest	Test according to instruction manual			
Castors	Free movement an locking			
Knurled screws	Tight fit			
Side rail	Safe locking/unlocking			
Lower leg rest	Latching			
Accessories (e.g. trapeze pole, trapeze)	Secure, Damage			
Signature of the tester:	Result of the test:			Date:

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BY THE OPERATOR

The operator of the nursing bed according to MPBetreibV § 4 is obligated with every new assembly, every maintenance and throughout the operation, to perform regular inspections, to ensure the safe condition of the nursing bed.

These tests are to be repeated during regular maintenance and depending on the use according to MPBetreibV § 4 and from the Directive of the German Employers Liability Insurance Association according to DGUV A3.

♦ Observe this sequence for the inspection according to DIN EN 62353:

I. Visual inspection

II. Electric Measurements

III. Function Test

♦ The performance of the function test as well as the rating and documentation of the test results can, according to MPBetreibV § 4 only be performed by experts with the appropriate knowledge, training and experience as well as the tools necessary to perform it according to the rules.

♦ The performance of electrical measurements according to DIN EN 62353 can, with availability of proper measuring equipment, also be performed by an electro-technically trained person [according to DGUV A3] with medical and appliance specific training.

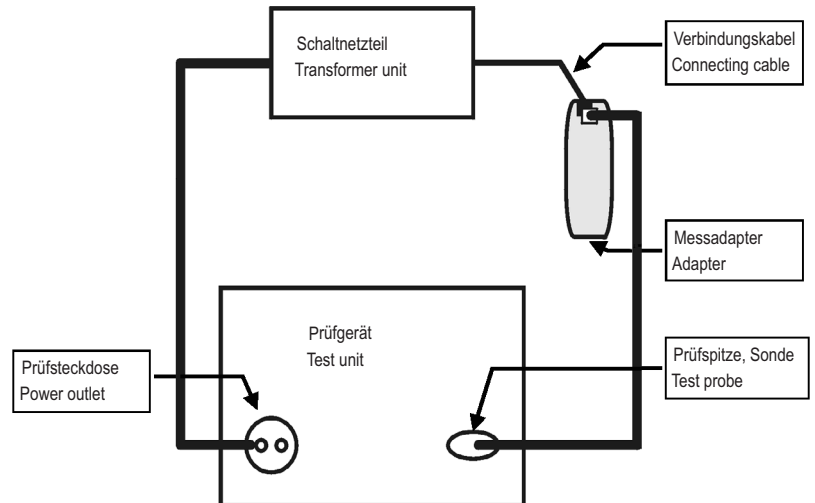
♦ The assessment and documentation of the test results can only be performed by an expert with the corresponding knowledge, training and experience.

♦ The testing of the transformer unit only requires measurement of the leakage current. The bed can therefore be used again once the transformer has been replaced with a new unit.

♦ BURMEIER offers transformer testing as a customer service. Send the transformer unit back to BURMEIER and receive a tested replacement unit. Contact us for further details; Address see page 43.

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Preparation:

- Unplug the power plug of the transformer unit from the wall socket.
- Unplug the connecting cable of the transformer unit from the connecting socket.
- Plug the plug of the connecting cable in the adapter (extra accessory; request from BURMEIER).
- Connect the adapter with the test probe of the test unit.
- Plug the transformer power plug into the power socket of the test unit.

Test method:

- Leakage current test: direct or differential according to DIN EN 62353
- Leakage current test according to the test unit manufacturers instructions.

Limit:

- Leakage current power less than 0.1 mA.

Test cycle:

We recommend a yearly visual inspection and performance check. If this check is OK, then it is sufficient to carry out an electrical test every ten years.

If the 2% error quota is shown to have been observed, the test cycle can be extended to a maximum of three years.

If damage or a function error is suspected take the nursing bed out of service immediately and disconnect it from the power until an exchange or repair can be made!

The test protocol draft on **pages 41 and 42** can be used.

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Customer / med. Facility / practice:						
Address:						
Carried out: <input type="checkbox"/> Repeat inspection			<input type="checkbox"/> Inspection prior initial operation			
<input type="checkbox"/> _____			<input type="checkbox"/> Inspection following repairs / servicing			
Equipment type: <input type="checkbox"/> Hospital bed <input checked="" type="checkbox"/> Nursing bed			Protection classification: <input type="checkbox"/> I <input checked="" type="checkbox"/> II			
Bed type: INOVIA II / INOVIA II 100			Inventory Number:			
Location:			Serial Number:			
Manufacturer: Burmeier GmbH & Co. KG			User-specific: none			
Testing equipment used (Type / Inventory Number):			1.			
MPG Classification: Class I			2.			
I. Visual Inspection				OK	Not OK	Defect Description
Visual inspection of the electrical components						
Stickers and type plates	Presence, legible					
Transformer unit housing	Damage					
Motor housing and lifting rods of motors	Correct position, Damage					
Motor bolt	Check that the lock washer is properly in place					
Hand switch	Damage, Foil intact					
Motor-, hand switch- and connecting cables	Damage, Routing					
Visual inspection of the mechanical components						
Stickers and type plates	Presence, legible					
Trapeze pole / socket	Damage, Deformation					
Bed frame	Damage, Deformation					
Castors	Damage					
Reclining surface	Damage, Deformation					
Wooden boards	Damage, Splinters					
Welded seams	Split welded seams					
Side rail	Damage, Splinters					
Knurled screws	Tight fit					
Wearing parts such as joints	Damage					
Locking ring of backrest/thigh rest	Check that the ring is properly in place					
II. Electrical measurement of the external transformer unit according to DIN EN 62353 (leakage current of equipment: direct or differential)						
Proceed as follow:				OK	Not OK	Defect Description
1. Plug in transformer unit in test socket of testing device						
2. Insert the plug of the connecting cable into the special adapter (extra accessory; request from BURMEIER)						
3. Connect the special adapter with the test probe of the test unit.						
4. Start procedures with the measuring instrument						
Limit Value	Value of first measurement	Current actual value				
Result: Bed SK II (Type B)	0,1 mA	mA	mA			

III. Performance check		OK	Not OK	Defect Description
Performance check of the electrical components				
Motion limit switching of the motors	Automatic cut-off			
Hand switch, blocking functions	Test according to instructions for use			
Drive motors	Noise level			
Drive motors	Test according to instructions for use			
Strain relief and security of routing the plug	Secure, Tight fit			
Strain relief for the transformer connecting cable	Secure, Tight fit			
Performance check of the mechanical components				
Joints and pivots	Smooth operation			
Castors	Drive and brake			
Emergency lowering of the backrest	Test according to instructions for use			
Side rail	Safe locking / unlocking			
Lower leg rest	Latching			
Accessories (e. g. trapeze pole, trapeze)	Secure, Damage, Suitability			
Result of the inspection:				
All values within permissible range: <input type="checkbox"/> yes <input type="checkbox"/> no		Next inspection date:		
Inspection passed: <input type="checkbox"/> yes <input type="checkbox"/> no				
If inspection was not passed: <input type="checkbox"/> Defective, do not use the bed! => Repair <input type="checkbox"/> Defective, do not use the bed! => Take out of Service <input type="checkbox"/> Bed does not meet the safety standards				
Test approval sticker applied: <input type="checkbox"/> yes <input type="checkbox"/> no				
Part of this inspection report:				
Comments:				
Inspected on:	Inspector:	Signature:		
Evaluated on:	Operator / Expert:	Signature:		

BURMEIER GmbH & Co KG
Pivitsheider Straße 270
D - 32791 LAGE
☎ + 49 - 05232 - 9841 - 0

Model	
Type	
Serie	

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Model	Bed model
Type	Item Number
Serial	Serial Number

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SPARE PARTS

The corresponding replacement parts can be obtained from BURMEIER by specifying the item number, the order number and the serial number. The necessary details are found on the type plate under the cross tubing of the reclining surface frame.

MANUFACTURER'S ADDRESS

In order to maintain operational safety and the right to claim under warranty, only original BURMEIER replacement parts may be used!

For ordering replacement parts, customer service requests and other questions, please contact:

Burmeier GmbH & Co. KG
(A subsidiary of the Stieglmeyer Group)

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REPLACEMENT OF ELECTRICAL COMPONENTS

Dangerous voltage, electric shock possible

- Before doing any work unplug the main power cable from the wall socket!
- Any work on the electrical equipment may only be performed by customer service, the motor manufacturer or qualified and authorized electricians under consideration of the applicable VDE rules and safety guidelines!

- Only dismantle the drive motors when the reclining surface is in the horizontal position, otherwise there is the risk of being injured by falling parts of the mattress base.
- All the drive components are maintenance-free and must not be opened. In the event of a malfunction, the corresponding components should always be replaced in full!

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Exchange of the Hand switch

- ☞ Unplug the power plug from the wall socket.
- ☞ Turn the plug cover on the drive unit to remove it.
- ☞ Unplug the plug at the end of the spiral cable of the hand switch from the connector of the drive unit.
- ☞ Lay the cable of the new hand switch so that it cannot be damaged by moving parts of the bed.
- ☞ Insert the plug at the end of the spiral cable of the new hand switch into the connector of the drive unit. Take care that the O-ring on the plug is not damaged, otherwise the connection may not be moisture-proof.
- ☞ Carry out a function test of the electrical adjustments after the exchange.

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TROUBLESHOOTING

The following table on page 45 is a guide to rectifying common malfunctions:

If a function fault is detected during operation, which the user cannot rectify with the use of the trouble-shooting table aid, the specialized staff for maintenance and repair of the appropriate owner is to be called.

The user may **under no circumstances** try to eliminate defects at the electrical equipment!

Dangerous voltage, electric shock possible!

Any work on the electrical equipment may only be performed by service personnel, the motor manufacturer or qualified and authorized electricians as required by the applicable VDE rules and safety guidelines!

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Trouble-shooting table aid

Problem	Possible causes	Solution
Hand switch / drive system not functioning	<ul style="list-style-type: none"> ■ Hand switch plug not properly connected ■ Transformer plug not inserted properly ■ Hand switch or drive system defective ■ Functions are blocked on hand switch 	<ul style="list-style-type: none"> ☞ Test the plug connection ☞ Press the transformer unit plug firmly into the connecting socket ☞ Inform your operator about necessary repairs ☞ Unlock functions (see page 24)
Green LED on transformer does not light up and drive systems do not function	<ul style="list-style-type: none"> ■ Power cable not properly connected ■ Wall socket without power ■ Transformer defective 	<ul style="list-style-type: none"> ☞ Plug in the power cable properly ☞ Test the wall socket and fuse box ☞ Exchange the transformer; inform your operator about any necessary repairs
Green LED on transformer lights up continuously, yellow LED does not light up when any hand switch button is pressed	<ul style="list-style-type: none"> ■ Hand switch is defective ■ Transformer plug not inserted properly ■ Functions are locked on hand switch 	<ul style="list-style-type: none"> ☞ Exchange the hand switch ☞ Test the plug connection ☞ Unlock the functions (see page 24)
Drives operate in one direction only	<ul style="list-style-type: none"> ■ Hand switch or drive system is defective 	<ul style="list-style-type: none"> ☞ Inform your operator about necessary repairs
Drives stop suddenly after prolonged adjustment times	<ul style="list-style-type: none"> ■ Heat switch in transformer is activated 	<ul style="list-style-type: none"> ☞ Pay attention minimum pause (see page 22) ☞ Exchange the transformer; inform your operator about any necessary repairs

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ACCESSORIES

Only the Original BURMEIER accessories may be used. If other accessories are used, we can not be liable for any resulting injuries, defects and dangers.

When using side rail, drip/I.V., etc. with electrically adjustable beds, please note:

Make sure that the arrangement of accessories does not produce any crush or shearing zones for the patient when the back and leg rests are adjusted. If this cannot be ensured, the user must safely prevent the patient from adjusting the back- and leg rest.

► To do so, attach the hand switch out of reach (e.g. at the foot end of the bed), or lock hand switch adjustments.

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REQUIREMENTS OF THE MATTRESS

Basic dimensions:
Length x Width
Thickness/Height
Volume density of foamed material
Compression load deflection
Norms to be observed:
DIN 13014
DIN 597 Section 1 and 2

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Assembled bed with side rails:

Reclining surface
External dimensions
Total weight
Safe load

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- ♦ Electrical height adjustment of the reclining surface from about 22 cm to 77 cm.
- ♦ Electrical adjustment of the backrest from 0° to about 70°.
- ♦ Electrical adjustment of the leg rest from 0° to about 35°.
- ♦ Electrical inclination of the reclining surface to the foot end from about 10°.
- ♦ Simultaneous adjustment of the back and thigh rest (Auto-contour).
- ♦ Moves on four castors. Each axle can be locked separately.

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INOVIA II

- ♦ Reclining surface of 200 x 90 cm, divided into four areas; Outside dimensions are about 219 x 105 cm.
- ♦ Side rails on both sides can be lowered.

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INOVIA II 100

- ♦ Reclining surface of 200 x 100 cm, divided into four areas; Outside dimensions are about 219 x 115 cm.
- ♦ Side rails on both sides can be lowered.

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Hand switch with locking function

Type
Protection

**Motors for reclining surface height
adjustment**

Type
Power / Size / Lift
Input Voltage
Operating Time
Protection

Drive unit for backrest and thigh rest

Type
Power (dynamic); backrest / thigh rest
Input Voltage
Output Volts
Operating Time
Protection
Classification, Protection class II, Type B, MPG classification Class I, not for use in explosive atmospheres.

Explanation of Icons:

- ♦ Unit with Type B application according to IEC 601-1
(Special protection against electric shock)
- ♦ Protection Class II device, double insulation
- ♦ Transformer with thermal fuse block
- ♦ Safety transformer in accordance with VDE 05510
- ♦ Caution! Consult instruction manual

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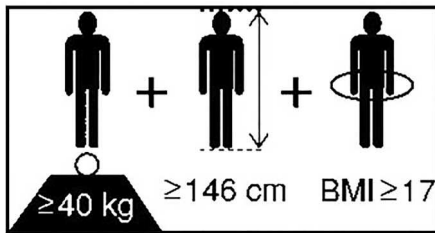
Noise level during motor adjustments

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The ambient conditions stated below must be observed:

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Storage temperature
Relative humidity

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Ambient temperature
Relative humidity
Air pressure

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Risk analysis for medical devices
Safety for electromedical equipment
Electromagnetic compatibility
Medical electrical equipment: with fitness for use
Medical electrical equipment – Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment
Particular requirements for the safety and essential performance of medical beds
Medical electrical equipment: Particular requirements for the basic safety and essential performance of medical beds

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♦ This nursing bed is - if electrically adjustable - classified as a commercially used electric appliance (b2b) in accordance with WEEE-Guidelines 2002/96/EG (Electronics Law).

♦ Exchanged electronic components (drives, control unit, hand switch, etc.) of these nursing beds are considered electronic scrap and must be disposed of according to the WEEE-Guidelines.

♦ With nursing beds that came into use after August 13, 2005 (see designation "Serial" on the type label at the head of the bed), the operator is required to not dispose of the electrical components in the municipal waste area, but to send them directly to the manufacturer.

BURMEIER and their service and sales partners will take these parts back.

♦ Our general terms and conditions apply for this redemption.

♦ No longer useable rechargeable batteries have to be disposed of according to regulations and do not belong in the house garbage.

♦ With all disposed components the owner must make sure that they are not infectious or contaminated.

♦ In the event that the nursing bed must be scrapped, the plastic and metal parts used must be disposed of separately, following the proper disposal procedure.

♦ Pneumatic springs are under high pressure! Before disposal they have to be depressurized. Please contact the manufacturer of the device on how to depressurize them (see type plate).

♦ Hydraulic units are made up of mostly metal and plastic. Before disposing according to manufacturer guidelines the oil has to be drained and disposed of properly.

♦ Contact your local community, disposal facility or our service department with any questions.

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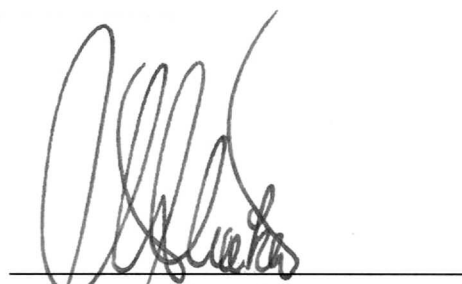
We,

Burmeier GmbH & Co. KG
Pivitsheider Strasse 270
32791 Lage / Lippe
Germany

hereby declare under sole responsibility as the manufacturer that the product model named below:

Nursing bed INOVIA II / INOVIA II 100

in the version submitted complies with the regulations of the EC Directive 93/42/EEC for Medical Devices, last amended by Directive 2007/47/EC dated 5 September 2007. It is categorised as a Class I active medical device.

The relevant technical documentation is kept by the manufacturer's safety representative.

To evaluate the conformity to the Directives, all applicable parts of the following standards were referred to:

complies with the regulations of EC Guidelines 93/42/EEG for medical products and fulfills all additional safety requirements by the Highest State Authority from May 22, 2001 for care beds.

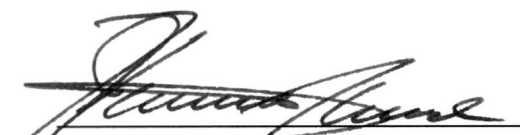
Harmonised standards:

- EN 14971 : 2013-04 Risk analysis for medical devices
- EN 60601-1 : 2007-07 Safety for electromedical equipment
- EN 60601-1-2 : 2007-12 Electromagnetic compatibility
- DIN EN 60601-1-6 : 2010-10 Medical electrical equipment: with fitness for use
- DIN EN 60601-1-11 : 2010-10 Medical electrical equipment – Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment
- DIN EN 60601-2-52 : 2010-12 Particular requirements for the safety and essential performance of medical beds


International standards:

- IEC 60601-1-2-52 : 2009-12 Medical electrical equipment: Particular requirements for the basic safety and essential performance of medical beds

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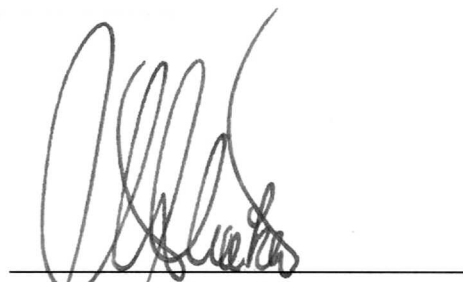No part of this manual may be reproduced with-
out the prior written permission of the publisher!

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All rights reserved.
Subject to technical changes!

Stand: **2017.01.06**

Updated: **2017.01.06**