



BURMEIER

Instruction Manual

Solo / Duo

Overbed Table and Side Table



	Model Number	Type
Solo	30.8191.1	218932
Duo	30.9191.2	218931



Date: 20/03/0217



Contents

1 Foreword	3
2 Safety Instructions	4
3 Use	4
3.1 Use for the Intended Purpose	4
3.2 Assembly:.....	5
3.3 Functions:.....	6
4 Care Instructions	6
5 Replacement parts	7
6 Technical Data	8
7 Disposal Instructions	8

1 Foreword

Dear Customer,

BURMEIER would like to thank you for the confidence you have placed in us and our products in deciding to purchase this bed and side table.

Each appliance has been tested by the manufacturer for safety and functionality and has left our factory in perfect condition.

This instruction manual informs you as the operator and your users about all the functions necessary to ensure ease of operation and safe handling of this product on a daily basis.

This instruction manual should therefore also be regarded as a practical reference book to be kept near the bed and side table and to hand at all times.

We wish you and your users every success in using this item and are confident that this product enables us to make an important contribution to achieving that.

Burmeier GmbH & Co. KG

2 Safety Instructions

Before using the appliance for the first time:

Read this instruction manual from start to finish to prevent any injury or damage resulting from incorrect operation.

Clean and disinfect the bed and side table before initial use and before each re-use.

Instruct the user in the safe operation of the bed and side table.

Each time before using a bed and side table, check that it is functional and that the bed and side table and the accessories are in perfect working order.

The maximum total load of the table is: 8 kg

The maximum central load of each table top is 8 kg. **Do not lean on the table!**

Always put the brake on the castors.

3 Use

3.1 USE FOR THE INTENDED PURPOSE


- The bed and side table was developed as a supporting solution for home care, for infirm, bedridden, disabled or frail persons. It is designed to support this care. It is used for storage and for helping with food intake.
- The bed and side table can be positioned on the floor, independent of the care bed. As the table top is height-adjustable, it can also be used above the mattress base of the care bed.
- The bed and table top can be used independently of those requiring care.
- This bed and side table may only be operated by persons who have received instruction in its safe operation.
- This bed and side table is suitable for repeated use. When re-using, pay attention to the necessary requirements:
 - Care instructions (see Section 4)

3.2 ASSEMBLY:


1. Screw the upright column with two wing nuts (1) onto the chassis.
The column must be positioned as seen in the photo – risk of tipping!
2. Place the table top in the desired position and screw tight using the wing screw (2).
3. Adjust the height by loosening the wing screw (3) and then fixing into position.
4. Castor brake: foot pedal (4) down.

Duo (optional)

5. Screw the fixed table top to the chassis using a wing nut (5).
- Ensure that all wing nuts are tightened properly!
 - Maximum central load per table top: 8 kg
 - Perform function test after installation!

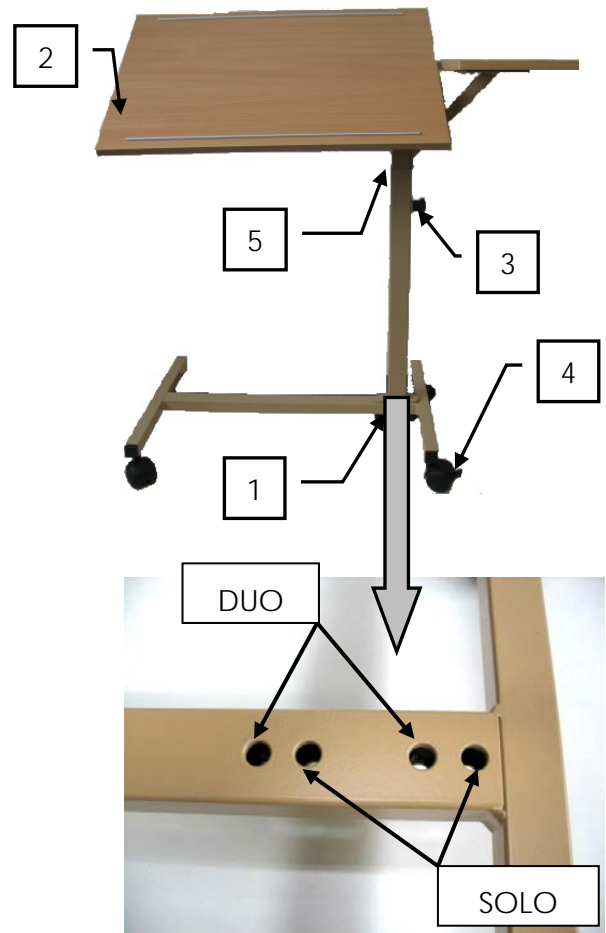

Positioning the upright stand on the chassis



Warning


- The table top can be pulled out to a maximum height of 111 cm. If raised any higher, there is a risk that the retractable tube is pulled out of the guide tube. There is a risk of tipping the table top

3.3 FUNCTIONS:



Maximum total load of the table: 8 kg

- Swivelling the table top. Make sure that the wing bolt is always tightened after adjusting.
- Adjusting the height of the table top
Make sure that the wing bolt is always tightened after adjusting.

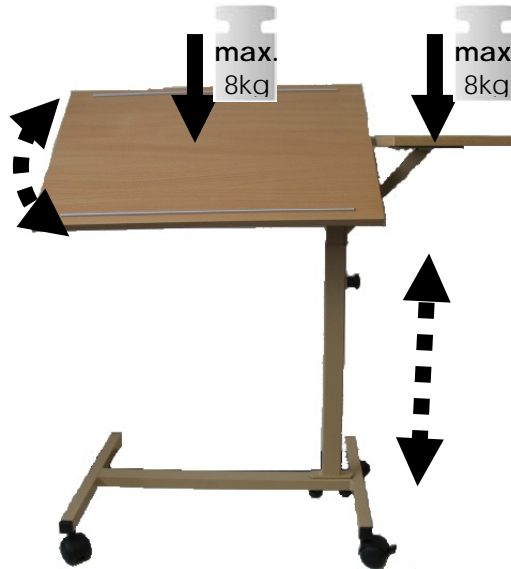


4 Care Instructions

Cleaning is the most important measure and requirement for a successful chemical disinfection. If used by the same patient, routine cleaning is generally sufficient in terms of hygiene. Disinfection of the bed and side table is only necessary when it has been visibly contaminated with infectious or potentially infectious materials.

Pay attention to the following recommendations to ensure that the bed and side table functions and usability are preserved as long as possible:

- The surfaces must be undamaged, any damage must be repaired immediately.
- We recommend (damp) wipe cleaning. When selecting cleaning agents, ensure that the ones chosen are mild (gentle to skin and surfaces) and environmentally friendly. A standard household cleaner can generally be used.
- The used materials are resistant to tested and approved disinfectants.
- For cleaning and care of the bed and side table, use a damp cloth with standard, mild household cleaner suitable for cleaning.
- The used materials are resistant to tested and approved disinfectants. For disinfection by wiping, the disinfectant agent used should be an appropriate and approved DGHM-listed disinfectant (in the recommended concentration). Agents and concentrations indicated in the list of the Robert Koch Institute (or the former Federal Health Office), may only be used in the event of an outbreak in accordance with the German Protection against Infection Act only by order of the responsible health authority.

- The table top of the bed and side table is made from high-quality ABS plastic. In the case of ABS plastics, colour changes to the surface may occur in conjunction with slightly oily liquids (depending on the pH value). This can result in unsightly discolourations and anomalies appearing on the surface. However, this does not affect the strength of the plastic material.
- Liquids that can cause such discolourations and are most commonly encountered are coffee, tea or even Kodan Forte (coloured), for example.
- For this reason, please observe the care instructions for ABS plastics:
- Do not allow coffee, tea and Kordan Forte stains to dry. Wipe up as soon as possible with a commercially available cleaning agent.

 Advice	Do not use scouring agents, stainless steel cleaning agents, abrasive cleaning agents or scouring pads. These substances can damage the surfaces.
--	---

5 Replacement parts

The relevant replacement parts are available from BURMEIER upon specifying the item number and serial number. The necessary details are found on the type plate located on the lifting column.

BURMEIER GmbH & Co KG Pivitsheider Straße 270 D - 32791 LAGE Fon: +49 - (0)5232 - 9841 - 0			max Stützlast 8 kg
Model	SOLO		
Type	218932		
Serie	05/12		
		Made in EU	

BURMEIER GmbH & Co KG Pivitsheider Straße 270 D - 32791 LAGE Fon: +49 - (0)5232 - 9841 - 0			max Stützlast 8 kg
Model	DUO		
Type	218931		
Serie	05/12		
		Made in EU	

In order to maintain operational reliability and the right to claim under warranty, only original BURMEIER replacement parts may be used!

Please refer to the general terms and conditions for our warranty conditions.

To order replacement parts, or make customer service requests or other queries, please contact:

BURMEIER GmbH & Co. KG

(A STIEGELMEYER Group company)

Pivitsheider Straße 270 · D-32791 Lage/Lippe

Tel.: +49 (0)52 32/98 41- 0 Fax: +49 (0)52 32/98 41- 41

Email: auftrags-zentrum@burmeier.de

6 Technical Data

	Solo	Duo
Dimensions: Depth / Length	40 cm/ 60 cm	40 cm / 81 cm
Height of table top:	79 cm to 111 cm	79 cm to 111 cm
Table top, swivelling:	60 x 40 cm	60 x 40 cm
Table top, fixed		20 x 40 cm
Swivelling the table top:	+60° /- 90°in 15° ratchet movements	+60° /- 90°in 15° ratchet movements
Max support load table top:	8 kg (table top centre)	8 kg (table top centre)
Castor set:	Double castors (braked) Ø 50 mm	Double castors (braked) Ø 50 mm
Chassis height:	9 cm	9 cm
Total weight:	10.5 kg	12.2 kg

7 Disposal Instructions

All packaging materials are be disposed of properly (German Green Dot System).

- If the unit is to be scrapped, the plastic, wooden and metallic parts must be separated and disposed of properly.

Please contact your local waste disposal company for this purpose.







Published by:

Burmeier GmbH & Co. KG
(A STIEGELMEYER Group company)

Pivitsheider Straße 270; • D - 32791 Lage/Lippe

Tel.: +49 (0)52 32/98 41- 0 • Fax: +49 (0)52 32/98 41- 41

www.burmeier.de

Email: auftrags-zentrum@burmeier.de



No part of this manual may be reproduced without the prior written permission of the publisher.

All rights reserved.

Subject to technical changes without notice.