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PREFACE

Dear customer,

Thank you for the confidence you have shown in BURMEIER.

The nursing bed ROYAL is factory-checked for electrical safety and function and has left our factory in faultless condition.

Please take time to read this manual carefully.

This applies especially to the assembly phase but also to daily use.

It is also a practical reference guide. Keep the instructions in a handy place.

These instructions provide information for the operator and user regarding the convenient handling and safe operation.

We wish you and your users much success in care-giving and we are convinced that our product will make a positive contribution.

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GENERAL NOTES

These instructions may describe functions and accessories which are not included with your model.

ROYAL Model:

Model Number: 50.6431.1

Wooden Reclining surface, 4 parts

Wooden side rails, 2 bars

Before putting the nursing bed into operation for the first time:

■ Read these instructions carefully in order to avoid damage or incorrect operation.

■ Follow the references for authorized mattresses according to DIN 13014 (see page 46).

■ Clean and disinfect the nursing bed before first use.

According to the "Medizinprodukte-Betreiberverordnung (MPBetreibV) § 2, before using the nursing bed, the user shall ensure that it is in proper working order and free of defects, and be aware of the instruction manual. This applies also to accessories.

The nursing bed ROYAL complies with all requirements of the guidelines 93/42/EWG for medical products. It is classified as a dynamic medical product of the class I in accordance with "Medizinproduktegesetz (German abbreviation: MPG § 13, Medical Products Act)."

The nursing bed ROYAL is checked by an independent examining institute. As with all technical, electrical devices improper handling can lead to damage and/or injury.

Observe your obligations as operator in accordance with medical devices "Medizinprodukte-Betreiberverordnung (German abbreviation: MPBetreibV, Operators of Medical Products Ordinance)" in order to ensure a dependable and safe operation of this medical device without endangering patients, users and third parties.

This manual contains safety guidelines, which must be observed. All persons who work with the nursing bed ROYAL must familiarize themselves with these instructions and follow the safety guidelines.

DEFINITIONS OF INVOLVED PERSONS

In this manual the following persons are involved:

Operator

Operators (e.g.: Medical Supply Houses, Dealers and representatives, health insurance companies) are natural persons or legal entities, who utilize the nursing bed ROYAL or authorize its use. The operator is responsible for instructing the user in the proper use of the unit.

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User

Users are persons who as a result of training and experience are authorized to assemble, perform maintenance and/or operate the nursing bed ROYAL. Furthermore, the user can recognize and avoid possible dangers, and judge the clinical condition of the patient.

Patient

In this manual a patient is an infirm or handicapped person who reclines on the nursing bed.

Specialist staff

Specialist staff are employees of the operator who are authorized as a result of their education and training to deliver, assemble, dismantle and transport the nursing bed. Furthermore, they are familiar with the regulations for cleaning and disinfecting the unit.

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SAFETY INSTRUCTIONS

The nursing bed ROYAL represents the latest technological development at the time of delivery and is checked by an independent examining institute.

■ Do not use the nursing bed ROYAL if it is damaged.

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Warning about injuries to persons

Dangerous voltage. Life threatening.

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General danger. Injury or life hazard.

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Warning about property damages

Possibility of damage to drive, material or environment.

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Other symbols

Useful tip. For easier operation or better understanding of the unit.

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The safety symbol used does not replace the text of the safety note. Therefore read the safety instructions and follow them exactly!

SAFETY INSTRUCTIONS FOR OPERATORS

- Remember your responsibilities in accordance with the MPBetreibV in order to ensure the safe operation of this unit without endangering patients, users and third parties.
 - Before first use, instruct every user in the safe operation of the nursing bed in accordance with these safety instructions, which must be supplied along with the unit.
 - According to MPBetreibV 9 the user shall be notified of the location of this instruction manual.
 - Call attention to the dangers of improper use of the unit, especially in regards to the electrical drives and side rails.
 - The unit should only be operated by properly trained personnel.
 - Make sure that representatives are also familiar with the operation of the nursing bed.
 - During long term use it is recommended that the nursing bed be inspected for function and visible damage at regular intervals (Recommendation: yearly) (see chapter 6).
 - Connect the bed directly to an electrical outlet. Do not use extension cords and/or multiple outlet extension cords.
 - When attaching other equipment (e.g. compressors for positioning systems) make sure that they are mounted and can function in a secure and safe manner.
 - Do not put multiple outlet electrical extensions under the nursing bed. Leaking liquids can cause a fire hazard.
 - Pay special attention to:
Securely locating of all wiring, cables, tubes, etc.
 - ▶ See page 9: *Intended use*
 - ▶ See page 20: *Safety note for the electric drive system*
- For further information please contact the manufacturer of the accessories or BURMEIER directly.
- Make sure that your personnel follow the safety instructions.

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SAFETY INSTRUCTIONS FOR USERS

- Let the operator instruct you in the safe use of the nursing bed.
- Before each use make sure that the nursing bed is in proper and faultless condition.
- Take care that no obstacles, such as furniture or slanted ceilings interfere with the adjustment functions.
- Pay attention that when using additional electrical components, such as patient lifts, reading lights or compressors for positioning systems that their electrical cords do not get entangled or damaged by the moving parts of the bed.
- Connect the bed directly to an electrical outlet. Do not use extension cords and/or multiple outlet extension cords.
- Make sure that when attaching other appliances (e.g. compressors for positioning systems) that a safe mounting and function is guaranteed.
- Do not put multiple electrical outlets under the nursing bed. Leaking liquids can be a fire hazard.
- Pay special attention to securely locating all wiring, cables, tubes, etc.
- Adjust the mattress base height whenever necessary, but at least once a day, to its upper or lowest height. This automatically equalises the two independent adjustment motors and results in a level horizontal mattress base.

For further information please contact the manufacturer of the accessories or BURMEIER directly.

■ Take the nursing bed out of operation if damage or a malfunction is suspected:

- ▶ Immediately unplug the electrical plug from the wall outlet;
- ▶ Clearly mark the nursing bed as "DEFECTIVE";
- ▶ Promptly notify the appropriate operator.

A checklist for assessing the proper condition is located in Chapter 6.

- Position the power cable in such a way that during normal operation of the nursing bed the cable will not be stretched, driven over or interfere with moving parts of the bed.
- Before any relocation of the nursing bed it is imperative that the power cable is pulled from the wall socket and be secured in the power cable holder.
- Check the power cable regularly for damage (abrasions, exposed wires, kinks, pressure marks, etc.) in particular:

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► after every larger mechanical strain (e.g.: rolling over the power cable with the nursing bed or with an equipment cart; after a strong pulling or bending load caused by the bed rolling away with the power cable still attached to the wall outlet);

► after relocating and before plugging in the power cable.

► during prolonged operating by the user.

■ Regularly inspect the cord grip of the power cable. It must be tightly screwed in.

■ Do not use the area under the nursing bed as a "parking spot" for any utensils.

■ Lower the reclining surface to its lowest position when leaving the patient unattended. This reduces the risk of injury to the patient getting in and out of the nursing bed.

■ Keep the hand switch safe from accidentally falling during non-use (Hang it on the hook). Take care that the cable is not damaged by the moving parts of the nursing bed.

■ To protect the patient and especially children from unintentionally adjusting the hand switch, place it beyond their reach (e.g. at the foot end) or lock the hand switch when:

► the patient is not able to control the bed safely or is unable to get out of dangerous positions without help;

► the patient could be endangered, because of unintentional adjustments to the drive motor;

► the side rails are up:
(There is a possible danger that limbs could be squeezed when adjusting the back and thigh rest);

► unsupervised children are in the room with the nursing bed.

■ Adjustment functions can only be performed by a trained person!

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PRODUCT DESCRIPTION

NORMAL OPERATING PROCEDURE

■ The nursing bed ROYAL, hereafter called bed, serves as a comfortable solution for positioning and easing the care of infirm, frail people in senior or nursing homes. Additionally it was developed as a comfortable solution for in-home care of infirm, handicapped or frail people. It will support this care.

■ The ROYAL care bed is designed for use in hospitals only in rooms of application group 0.

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■ This bed can be used for care under the instructions of a physician and can serve in diagnosing, treatment and observation of a patient. It is therefore equipped with a locking mechanism for the electrical adjustment functions.

■ This bed has no special connector possibilities for a potential equalization. Consider this before combining with additional electrically operated (medical) devices.

Further references about additionally applicable precautions can be found:

- ▶ In the instructions for the additional electrically operated devices (e.g. compressors for positioning systems, infusion pumps, nutrition probe ...)
- ▶ In the Standard DIN EN 60601-1-1:2002 (Safety of medical electrical systems)
- ▶ In the Standard VDE 0107:1994 (High current systems in hospitals)

■ The maximum long-term load for this bed is 175 kg (patient and accessories). Under this load, the bed can be operated without restriction.

■ The permitted weight of the patient depends on the total weight of the accessories attached at the same time (e.g. respirators, infusions,...)

Example:

Weight of accessories (with mattress):	Max. permitted weight of patient
10 kg	165 kg
40 kg	135 kg

▶ Observe, especially with patients in poor health condition, the safety tips in chapter 4.

■ This bed may only be operated by trained personnel.

■ This bed is useful for multiple re-use. Bear in mind the necessary requirements:

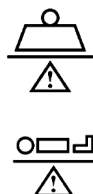
- ▶ See chapter 5;
- ▶ See chapter 6.

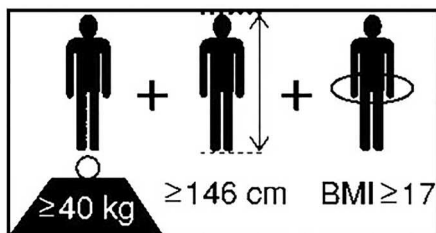
■ While the patient also is lying in bed, it is allowed to drive the bed within the building.

This bed may only be operated under the operating instructions laid out in this instruction manual.

This bed must not be altered in any way. Any other use is deemed to be inappropriate.

Sale of this product in North America, especially in the United States of America, is prohibited. Distribution and use of the nursing bed, including distribution and use by third parties in these markets is forbidden by the Manufacturer.

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CONTRAINDICATIONS

■ In case of using safety sides this bed is only suitable for patients who do not fall below the following minimum body size/weight:

- ▶ Height: 146 cm
- ▶ Weight: 40 kg
- ▶ Body Mass Index „BMI“: 17

Calculation of BMI =

Weight of patient (kg) / Height of patient (m)²

Example a

41 kg / (1.5 m x 1.5 m) = 18.2 = **OK!**

Example b

35 kg / (1.5 m x 1.5 m) = 15.6 = **Not ok!**

■ Owing to the smaller limbs of patients with lower body measurements/weight, there is an increased risk of entrapment between the open spaces of the safety sides when safety side systems are used.

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STRUCTURAL DESIGN

The bed is shipped disassembled. It can be transported without problems even in apartment homes. It consists of two end boards (head- and foot end) each of which consists of a tubular frame with fitted wooden boards; four side rail beams and a trapeze pole with trapeze handle. The bed stands on four sinkable and steerable castors.

Reclining surface frame

The reclining surface frame has four sections: a movable backrest, a firm middle section, a moveable thigh rest and lower leg rest. The back- and thigh rest can be adjusted with the aid of electric motors. The horizontal height of the reclining surface can be adjusted and inclined to a low foot end position. All adjustment functions are controlled by a hand switch.

Side Rails

The bed has side rail beams on both sides which can be raised as a barrier or lowered when not needed. This safeguards the patient from accidentally falling out of the bed.

Electric adjustment system

The electrical adjustment system of this bed is error protected, flame-retardant (V0) and consists of:

■ the central drive unit with the drive motors for the back and thigh rest. For the safety of patient and user, safety low-voltage of 24 volt is produced by means of a transformer. All drive motors and the hand switch are hooked up to the central drive unit by plug-in connections and work with the 24 volt safety low-voltage;

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- two drive motors for the height adjustment of the reclining surface;
- a hand switch with a sturdy hook;
- a separate locking box (accessory). The user can lock certain adjustments of the hand switch, if the bad clinical condition of the patient requires it.

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DESCRIPTION OF MATERIALS

The bed is mostly constructed out of steel sections, their surfaces have been polyester-powder-coated, or with a metallic coating of zinc and chrome. The end boards, the side rail beams and the spring wood slats consist of wood and derived timber products whose surface has been sealed.

All surfaces are harmless to skin contact.

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Beipack: Montagematerial

ASSEMBLY AND START UP PROCEDURE

The assembly is taken care of at the destination by qualified personnel supplied by the operator.

The bed ships in 5 cartons:

Carton 1: Reclining surface frame including drive motors (pre-assembled); Trapeze pole with trapeze

Carton 2: Chassis head section with drive motor

Carton 3: Chassis foot section with drive motor

Carton 4: 4 side rail beams; 2 side boards

Carton 5: Wooden boards, head and foot section

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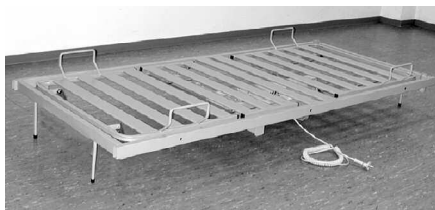
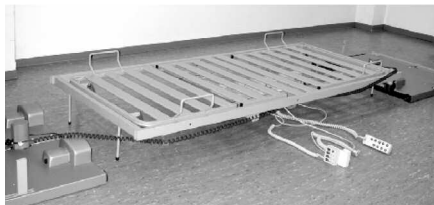
ASSEMBLY

PREPARATION

☞ Position the reclining surface frame [19] flat on the floor. Top side up.

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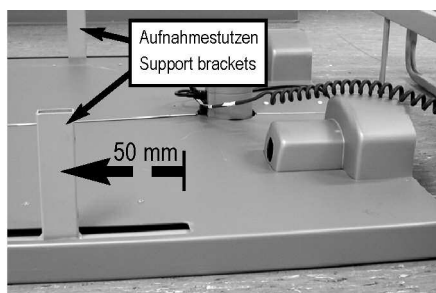
☞ Position the chassis frames [4] + [13] flat on the floor. The castors point upward.

☞ Connect both drive motors of the end boards to the drive unit. (Electrical connection see page 16).

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THE END BOARDS

Upon delivery, the support brackets are already extended to 50 mm. If a disassembled bed is reassembled, the support brackets must be extended to 50 mm before further assembly (see page 22).

For further assembly unplug the power plug from the wall.

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ASSEMBLY OF THE RECLINING SURFACE FRAME

Start the assembly at the head section.

Join the tubular frame/head section [13] with the reclining surface frame [19] together.

☞ Lift the reclining surface frame at the head end and push the two support brackets from the chassis/head section [13] into the tubes of the reclining surface frame [19] to the limit.

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☞ Screw the reclining surface frame firmly to the support brackets.

For this use two Allen screws M 8 x 20 mm on each side.

☞ Tighten the 4 Allen screws firmly with a size 5 Allen wrench.

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☞ Repeat the process with the chassis/foot section [12].

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ASSEMBLY OF THE SIDE BOARDS

☞ Raise the reclining surface to the middle horizontal position (see page 22).

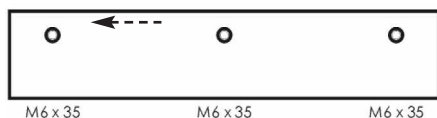
☞ Raise the back and thigh rests [10] + [7] half way. Lift the lower leg rest [6] to the horizontal position (see page 26).

Notice: The two side boards are identical.

The castors must be locked.

☞ Screw the side boards [17] to both sides of the reclining surface frame with 3 Phillips head screws respectively; fit the middlest screw first.

Take care, that the round edge of the side board point upward.

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ASSEMBLY OF THE WOODEN BOARDS

Notice: The wooden boards for the head and foot section are identical.

☞ Finally put the wooden board/head section [14] and the wooden board/foot section [5] respectively, on the tubular frame head and foot section [13] + [4].

Insert the wooden boards all the way.

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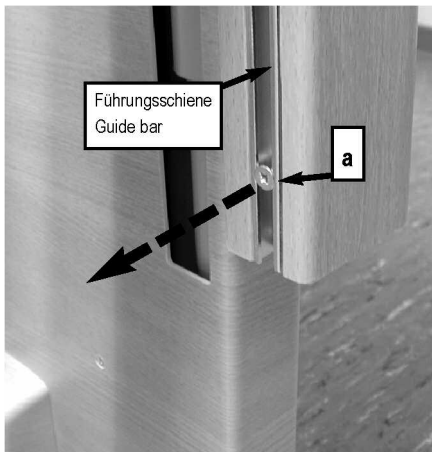
☞ Connect both wooden boards [5] + [14] on the top of both sides with phillips head screws M6 x 30 mm to the tubular frames [4] + [13]

☞ Cover all 4 drill holes with protective caps.

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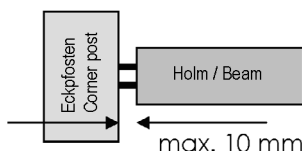
ASSEMBLY OF THE SIDE RAIL BEAMS

On all four corners of the bed you will find a guide bar embedded in the wooden boards [5] + [14] for the side rail beams [8]. In them runs a plastic slide with 4 metal pins. The metal pins hook into the oblong holes of the face of the 4 side rail beams. The side rail beams can be raised or lowered, as needed.

Start with the end board/head section:

☞ Remove the lower securing screws [a] from all guide bars.

☞ Push one of the plastic slides [b] from the bottom into the guide bar of the head section.

The wedge-shaped tip of the plastic slide must point upward.

☞ Push the plastic slide upwards in the guide bar, until it locks into ok place.

☞ Now stick the side rail beams onto the metal pins.

The top and bottom of the side rail beams are flattened and rounded respectively. The round side of the bars must point upward.

Now go to the end board/foot section.

☞ Insert one more plastic slide in the 2 ends of the 2 side rail beams.

The wedge-shaped tip of the plastic slide must point upward.

☞ Insert the plastic slider into the guide bar, then push it upwards until it locks into place.

☞ Mount the two other side rail beams on the opposite side of the bed the same way.

☞ Secure the security screws [a] into guide bars.

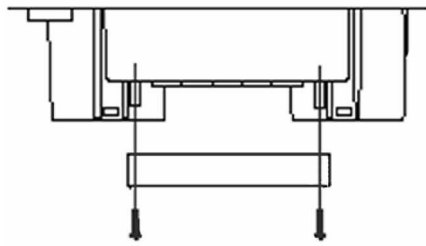
☞ Check the remaining distance between the end of the rail and the guide bar: max. 10 mm. If necessary decrease the distance by further inserting the support brackets in the reclining surface frame (see page 13).

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ELECTRICAL CONNECTION

Before connecting the cables all packaging materials must be removed.

The central drive unit supplies the drive motors, through a spiral cable, with the needed voltage. The plugs, at the end of the spiral cables, must be plugged into the central drive unit.

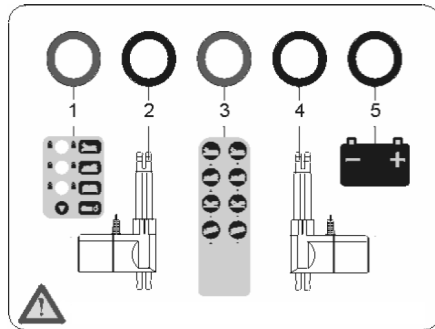
☞ Carefully remove the covering strip with a screwdriver.

DRIVE UNIT PLUGS

The spiral cables leading to each drive motor are marked in the following way at the central drive unit:

- 1 Locking box or bridge plug
- 2 End board / Foot motor
- 3 Hand switch
- 4 End board / Head motor
- 5 Not enabled (blind cap)

Before the plug can be inserted into the central drive unit, the covering strip has to be removed.

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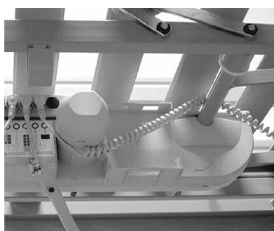
Take special care in assigning the drive motors for the height adjustments. These must not be interchanged. Spiral cable number 4 leads to the drive motor for the end board (head section). Spiral cable number 2 leads to the drive motor for the end board (foot section). Only one lower foot position should be possible.

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☞ Place the drive motor cables and the hand switch cable under the reclining surface frame.

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
Make sure that no cable is damaged, that there is no loop in the cables and that the cables are not crushed by moving parts.

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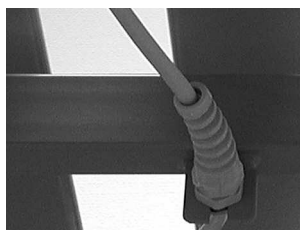When exchanging single electrical components, always make sure that their plugs have been plugged back into the central drive unit until they lock and the plug covering strips have been re-affixed. This will ensure a tight fit and a proper operation.

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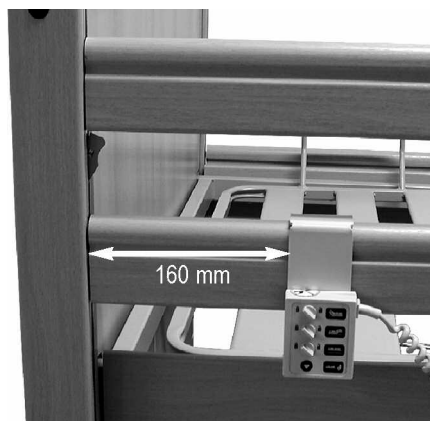
STRAIN RELIEF OF THE POWER CABLE

Underneath the reclining surface frame is the strain relief for the power cable.

☞ Make sure that the power cable is hooked tightly into the strain relief.

Special care is to be taken when positioning the power cable. Make sure that the cable is not damaged and there is no loop in it! When moving the bed around, take care that the castors do not run over the power cable! When adjusting the bed the power cable should not get caught between moving parts!

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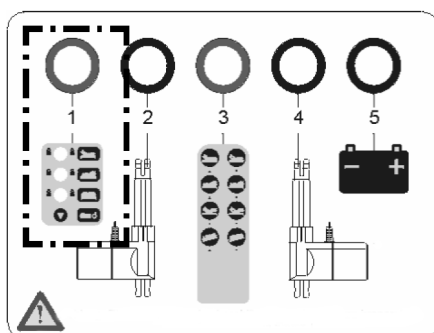
ASSEMBLY OF THE LOCKING BOX

The locking box is accessory.

The locking box should be connected on the lower side rail beam, 16 cm away of the foot end.

☞ Screw the mounting for the locking box onto the side rail beams with Phillips screws M 3.5 x 16 mm.

☞ Slide the locking box onto the mounting.

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☞ Place the cable of the locking box over the round cross beam tube to which the drive unit is connected.

☞ Affix the cable with cable ties to the square cross beam tube.

☞ Remove the cover strip from the drive unit.

☞ Remove the bridge plug from the connector 1 and insert the plug of the locking box.

☞ Re-attach the cover strip to the drive unit.

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LOCATION REQUIREMENTS

■ There must be enough space available to be able to adjust the bed. No furniture, window sills and the like should be in the way.

■ The space under the bed must remain clear.

■ Check before use on parquet flooring, that the castors do not cause discoloration of the varnish. Use on ceramic tiles, carpet, laminate and linoleum is harmless.

BURMEIER is not liable for any damage to floors due to daily operation.

■ A properly installed 230 volt power outlet must be available, near the bed.

■ Take care when attaching further accessory equipment (e.g. compressors for positioning systems, etc.) that their attachment is safe and their functionality is not impaired. Especially important is hereby also the safe placement of all movable connecting cables, tubes, etc. If you have any further questions or need clarification contact the manufacturer of the accessory equipment or BURMEIER.

Observe the following points when setting up the bed to minimize possible fire hazard.

Instruct the user and the authorized personnel in these points!

■ If possible use only fire-retardant mattresses and bed sheets.

■ Only use appropriate firm mattresses according to DIN 13014. This mattress should be inflammable according to DIN 597 Section 1 and 2.

■ Avoid smoking in bed because of the danger of fire.

■ Use only technically faultless appliances (e.g. heating blankets) and other electronic devices (e.g. lamps, radios).

■ Take care that these devices are only used for the purpose intended and that they cannot accidentally get on top of or underneath the bed sheets (Fire hazard because of overheating).

■ Absolutely avoid the use of extension cords or multi outlet extension cords under the bed (Leaking liquids can cause a fire hazard).

■ Connect the bed directly to an electrical outlet. Do not use extension cords and/or multiple outlet extension cords.

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REACHING OPERATIONAL READINESS

- After successful assembly, complete an inspection according to chapter 6.
- Clean and disinfect the bed before first time use and before every further use according to chapter 5.

After successfully completing and heeding all steps on page 12 through 19 the bed is now operationally ready.

DISASSEMBLY

- ☞ Remove the trapeze pole.
- ☞ Lower the bed into the middle horizontal position.
- ☞ Pull the power plug out of the socket.
- ☞ Remove the security screws in the guide bars.
- ☞ Remove all side rail beams consecutively.
- ☞ Insert the plastic slide back into the guide bar.
- ☞ Unplug the plugs and strain relief from the drive motors on the head and foot sections.
- ☞ Remove the stop plate below the guide bars.
- ☞ Loosen the screws and raise both wooden boards from the tubular frames of end boards one by one.
- ☞ Loosen the Allen screws from the reclining surface frame on the foot section, to remove the tubular frame from the foot section; next loosen the Allen screws from the reclining surface frame on the head section, to remove the tubular frame from the head section.

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SPECIAL SAFETY INSTRUCTIONS FOR THE DRIVE UNIT

- Take care when adjusting the bed that no limbs of patients, users or any other persons, especially children, can be jammed or injured by getting under the movable parts or the bed frame.
- For the safety of the patient, lock the hand switch operation of adjustments to the back and thigh rests to avoid unwanted adjustments when the side rail beams are up. (Danger of getting limbs trapped when adjusting the back- and thigh rests).
- When using accessories with the bed it is necessary to: make sure that the arrangement of these accessories will not cause any places that might squeeze or hurt the patient. If this is not the case the operator must safely disable the adjustment function of the back- and thigh rests.
 - ▶ To do so place the hand switch out of the patient's reach (e.g. foot section) or lock the adjustment functions on the hand switch by the locking box.
- Make sure that the power and hand switch cable cannot be jammed or damaged in other ways.
- Ensure that only hand switches that are in perfect condition are used. Faulty hand switches (e.g. with torn or indented foil) can result in spontaneous operation of the bed and can thus endanger the patient.
- Adjust the mattress base height whenever necessary, but at least once a day, to its upper or lowest height. This automatically equalises the two independent adjustment motors and results in a level horizontal mattress base.

- Before every relocation of the bed, always make sure that you have unplugged it from the mains supply.
- Each time before moving the bed, ensure that the mains cable will not be stretched, driven over or damaged in any other way. When moving the bed, the mains cable must be stowed in the designated mains cable holder.
- Make sure that when adjusting the bed no obstacles such as furniture or slanted ceilings are in the way. This will prevent damage.

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♦ An electronic overload detector switches off the central drive unit, when the load is too great. After clearing the overload and pressing the hand switch again the central control unit will work again.

♦ A continuous run time of 2 minutes should not be exceeded. Afterwards a pause of no less than 18 minutes should be observed. Another possibility: 1 minute continuous run time and 9 minutes pause.

♦ If the maximum continuous run time is not heeded (e.g. by constantly "playing" with the hand switch) a thermal safety component switches off the power for safety reasons. Depending on how large the over-load was, it can take several minutes before any more adjustments can be made.

♦ The adjustable range of every function is electrically/mechanically possible only within a predetermined range.

♦ Electrical disturbances affecting other nearby electronic appliances (e.g. crackling on the radio) cannot be excluded, despite adherence to all prescribed regulations. In such rare circumstances increase the distance between the appliances, do not use the same power socket or switch the affected appliance off for a while.

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HAND SWITCH

The electrical adjustments can be made using the hand switch [9].

The adjustable range of every function is electrically/mechanically possible only within a predetermined range.

An orange LED in the hand switch indicates the operational readiness of the bed.

Locking mechanisms have been built into the hand switch for safety reasons. For the safety of the patient, **all** adjustment functions can be locked at the hand switch, if, from a clinical point of view and with the approval of the attending physician, the condition of the patient warrants it.

There is an interaction between a locking box and the hand switch. **Certain** adjustment functions can only be performed with the hand switch, if they have been unlocked on the locking box.

♦ The hand switch, with its elastic hook can be hooked onto the bed.

♦ The spiral cable allows for greater mobility.

♦ The hand switch is water-tight and washable (IP 66).

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OPERATING THE HAND SWITCH

- Train the patient in the functions of the hand switch.
- The drive motors will run as long as the button is held down. A green LED indicates that a motor is running.
- A drive motor can only run, if the hand switch is unlocked or the corresponding adjustment function has been unlocked on the locking box.
- The adjustment functions, except for the lowered foot position, can be used in both directions.
- Basically the functions of the buttons are as shown:
 - raise
 - lower

Adjusting the backrest

Push this button if you want to adjust the backrest angle.

Adjusting the thigh rest

Push this button to adjust the thigh rest angle.

Setting up the Auto-contour

Press this button if you want to adjust the angles of the back and thigh rest at the same time. This prevents the patient sliding to the foot end.

Adjusting the reclining surface height

Push this button if you want to adjust the height of the reclining surface.

If the bed is in the lowered foot position, it will automatically be adjusted to horizontal level, when it is readjusted to the lowest or highest position.

Adjusting to the lowered foot position

Push this button if you want to adjust to the lowered foot position. With this button ➤ only the function "Down" is possible.

If the bed is in the lowered foot position, it will automatically be adjusted to horizontal level, when it is readjusted to the lowest or highest position.

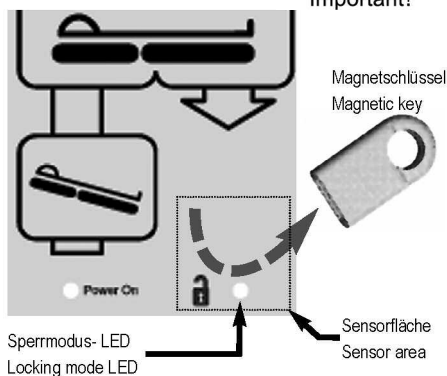
Use the "Height of the mattress" button for this adjustment.

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LOCKING FUNCTION

The magnetic locking key is delivered attached to the bed with a cable tie on the hand switch. The locking key is not intended for use by the patient. It must be removed from the hand switch. The operator or a person authorized by the physician should keep the **locking key**.

Sensor area for magnetic key

The hand switch should not be kept near magnetic objects or strong magnetic fields. Otherwise the locking functions could be activated or released.

After a power failure or location change of the bed, the locking function automatically reactivates.

Pass the magnet key once over the sensor area. The Mode LED must show green to perform adjustments.

If the locking mode LED is **not lit**, all adjustment functions are locked.

If the locking mode LED **lights up green**, all adjustment functions are unlocked.

If the locking mode LED **lights up orange**, all adjustment function are unlocked for a period of 40 seconds. After any adjustments within this time period, the locking function is reactivated automatically.

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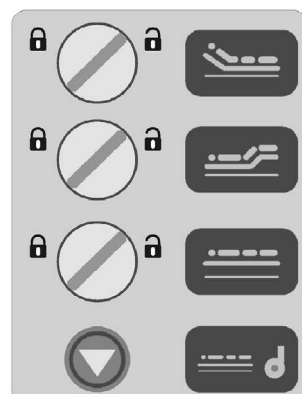
LOCKING BOX (ACCESSORY)

Only the operator is authorized to use the locking box!

In case the clinical condition of the patient is so critical that any adjustment of the beds by means of hand switch causes danger, the operator has to disable the bed immediately. The nursing bed remains in the position it had at the time of disconnecting.

Do not turn the locking key with force over the stop of the lock! The lock or the whole hand switch can be damaged.

When a knob of the locking box is turned to the right, the drive is unlocked.

When a knob of the locking box is turned to the left, the drive is locked.

When a key for any adjustment is pressed on the hand switch even though the drive is locked at the locking box, a signal tone will sound.

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Locking of the back rest

Turn the knob to the left. When the auto-contour switch is pressed on the hand switch, only the thigh rest will be adjusted.

Locking the thigh rest

Turn the knob to the left. When the auto-contour switch is pressed on the hand switch, only the back rest will be adjusted.

Locking the reclining surface height

Turn the knob to the left. The foot adjustment cannot be activated with the hand switch.

Extending the castors

Push this button to extend the castors in order to move the bed about. The knob "Locking of the reclining surface height" must indicate .

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CASTORS

The bed stands on four steerable and sinkable castors [15], which are contained in a housing. The castors must be extended whenever the bed is moved.

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Moving of the bed

Release the "Locking of the reclining surface" on the lock box.

☞ Swipe the magnetic key over the the sensor area.

The locking mode LED shows **green**.

☞ Push the button  until the lowest reclining surface height has been reached.

☞ Swipe the magnetic key over the sensor area once more.

The locking mode LED shows **orange**.

☞ Push the  button again until the castors have been extended.

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Alternatively the castors can also be extended by pushing the button  on the locking box.

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Setting the bed down

☞ Swipe the magnetic key over the the sensor area.

The locking mode LED shows **green**.

☞ Push the button  until the castors have been completely retracted.

☞ Make sure the bed stands firmly on the floor.

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☞ Make sure the power cable is secured in the provided power cable holder, before moving the bed.

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POWER CABLE HOLDER

The power cable holder is located sideways on the reclining surface frame near the foot section.

■ The power cable must be secured in the power cable holder provided, before moving the bed, otherwise it might be damaged by stretching, by running over and squeezing.

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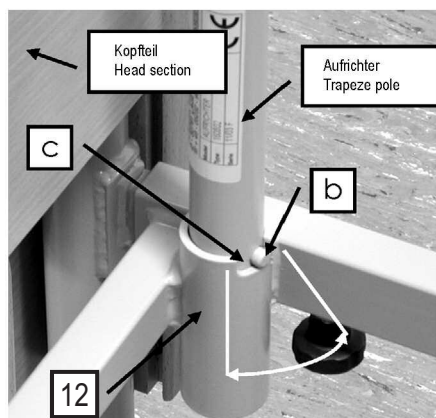
TRAPEZE POLE

■ The maximum load of the trapeze pole is 75 kg at the front end.

The trapeze pole attached to the bed makes it easier for the patient to get into and out of bed.

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In both corners of the head section of the reclining surface frame are two trapeze pole sockets [12]. The trapeze pole socket has a notch at the top [c], which together with the metal pin [b] restricts the swivel range of the trapeze pole. The trapeze pole should be installed on the side where the patient gets in and out of bed.

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Attaching

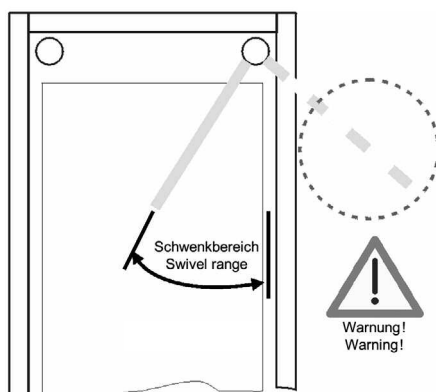
☞ Insert the trapeze pole [2] into the socket. The metal pin [b] must be in the notch [c] of the socket.

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Removal

☞ Pull the trapeze pole straight up out of the socket.

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Trapeze pole swivel range

Do not swivel the trapeze pole outside of the bed. There is a danger that through the pull, the bed will tip over.

The metal pin must always be in the notch of the socket.

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TRAPEZE

A trapeze [1] (grab handle) can be attached to the trapeze pole. The patient can hold on to the trapeze to make it easier to change to a different position.

Regularly check the trapeze at the strap for any damage (see Chapter 6). A damaged trapeze or strap must immediately be exchanged.

Durability

There is a date clock on the trapeze. With normal use the trapeze will last for at least five years. Afterwards a sight check is to be performed periodically to determine if further use is permissible.

Adjustable range of the trapeze

The height of the trapeze can be adjusted by means of the strap to accommodate a range of about 55 cm to 70 cm (measuring from the top of the mattress).

☞ Push the loop of the trapeze [1] over the first stud of the patient grab handle pole.

☞ Test by tightly pulling the trapeze downward that it is secure.

Useful Tip: The maximum lifting load of the trapeze pole is 75 kg at the front end.

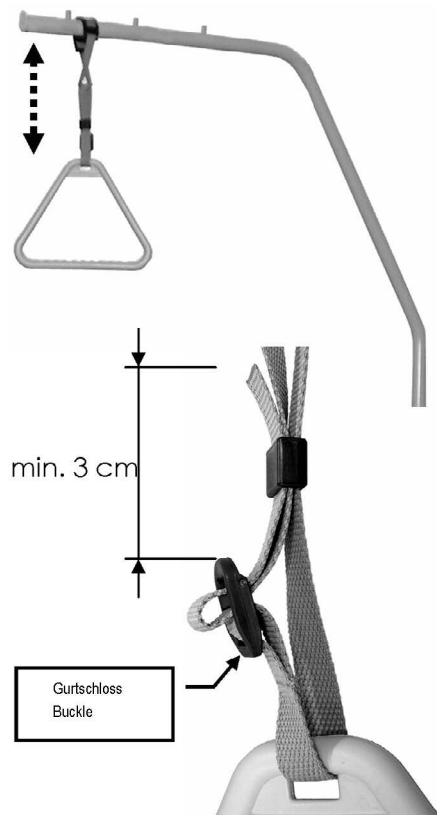
■ The height of the trapeze can be adjusted by means of the strap.

■ Make sure the the strap is fed correctly through the buckle.

■ Make sure that the end of the strap is at least 3 cm above the buckle.

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LOWER LEG REST

When the thigh rest is fixed, the lower leg rest [6] can be individually adjusted. For this reason there are two adjusting ratchets, which will lock the lower leg rest into position.

Raising

☞ Raise the lower leg rest on the foot end - not with the mattress brackets - up to the desired level. The lower leg rest will automatically snap into place.

Lowering

☞ First lift the lower leg rest all the way to the stopping point.

☞ Slowly lower the lower leg rest.

There is a potential for injury if the lower leg rest falls straight down.

If you lower the thigh rest, the lower leg rest will be lowered automatically.

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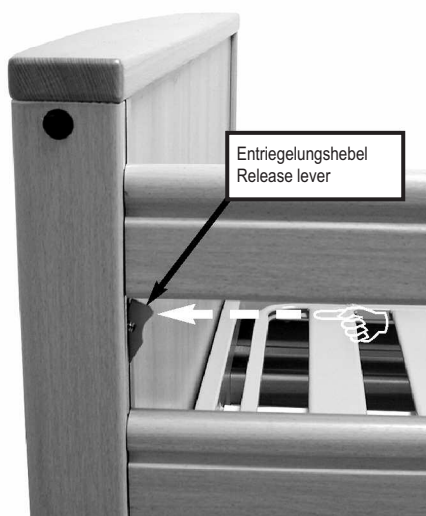
SIDE RAILS

Side rails conveniently prevent the patient from falling out of bed accidentally. They do not, however, prevent a patient from leaving the bed.

Engaging

☞ Pull the side rail beams [8] on one end after the other until they lock in its highest position. You should not be able to move them up or down.

☞ Check that the side rail beams are locked into place by pushing on them from above.

Disengaging

☞ Lift the side rail beams slightly.

☞ Push the upper release lever [21] and slowly lower the side rail beams.

Special safety notes for use of side rail beams:

■ Only use safely lockable, technically perfect, undamaged side rail beams with proper gap dimensions

■ Make certain that only suitable Original BURMEIER side rail beams are used, which have been approved for use with the bed model you are using or that are already integrated in the bed.

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■ Before using side rail beams evaluate and take into account the clinical condition and any special problems of the patient

► If, for example, the patient is very bewildered or very anxious, forgo the side rails and use alternative safety measures such as a restraining belt or restraint sheets.

► For especially small, frail patients, additional protective measures for reducing the distance between bars on the side rails may be necessary. In these cases use, for example side rail foam covers (accessories, see chapter 8), retaining belts, etc.

This will effectively ensure patient safety by reducing the risk of becoming trapped or slipping through.

■ Only use appropriate, firm mattresses accord to DIN 13014 with a height of 10 to a maximum of 18 cm. Further, this mattress should be fire resistant according to DIN 597 Section 1 and 2.

■ If elevated patient positioning systems are employed, such as antidecubitus mattresses, an effective side rail height of at least 22 cm above the unweighted mattresses must be ensured. If this requirement is not met, the operator must take suitable measures to safeguard the patient. This can be done by using an attachable bar for side rails (Accessory).

■ The mechanical parts on the bed frame and the side rails which serve to protect the patient must be regularly examined for damage. This should not only happen before the installation of the side rail but also during use in shorter intervals (at least before every new assignment).

■ To safeguard the patient against unintentional adjustments, always lock the electrical adjustment of the back and thigh rests on electrical beds when the side rails are raised. Otherwise there is a risk of trapping limbs when the back and thigh rest are adjusted.

■ Before lowering the side rail beams make sure that the patient's limbs are clear of the side rail beams. This way you can avoid injuring the patient.

If these safety warnings are not observed there is a danger for the patient of being injured by the side rails, of falling out of bed because of too large a distance between the bars or because of damage, the use of unsuitable side rails or the incomplete locking of the side rail.

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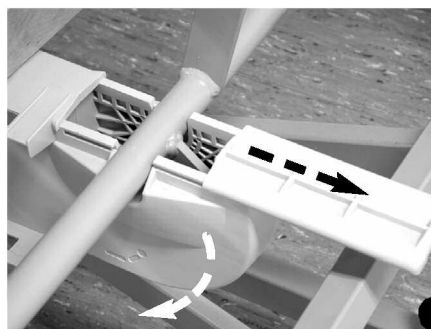
EMERGENCY LOWERING OF THE BACK REST

The upright back rest can be lowered manually in case of a power outage or failure of the power supply or the electrical drive systems.

In this case two users are absolutely necessary!

Disregarding these safety guidelines and instructions may cause the backrest to fall out of control, which could lead to serious injuries for the user and the patient!

Emergency lowering may only be carried out in the case of extreme emergency and by users who have a complete command of the procedures below.

We strongly advise you to practice emergency lowering of the backrest under normal conditions so that in case of an emergency you will be able to react quickly and properly.

☞ Before the emergency lowering, ease the load on the backrest.

☞ The first user raised the back rest slightly by the outside edge of the head section and holds it in this position.

☞ The second user now removes the cover slide on the side of the drive unit that drives the back rest.

☞ Swing the drive unit down and away.

The drive motor is now separated from the back rest.

☞ Now the first user carefully lowers the back rest.

If the back rest falls out of control, the patient and the second user may be injured!

☞ To return the bed to its original setup:

Swing up the drive unit and let the drive shaft click into place, now replace the cover slide.

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CLEANING AND DISINFECTING

GENERAL

Cleaning is the most important requirement for a successful chemical disinfection.

Generally a routine cleaning of the bed, if used by the same patient, is hygienically sufficient.

Disinfection of the bed frame is only necessary in case of visible contamination with infectious or potentially infectious material (blood, stool, pus) or by presence of an infectious disease under the direction of a physician.

The bed must be cleaned and washed down with disinfectant in case of a patient change!

Before beginning the cleaning, please note:

- Unplug the power cable from the wall socket and store it in such a way that it does not come into excessive contact with water or other cleaning solutions (store in plastic bag).
- Make sure all plugs at the central drive unit and the drive motors are properly connected.
- None of the electrical components must show any visible damage; otherwise water or cleaning solution could penetrate. This could lead to malfunction or damage to the electrical components.
- Before operating the bed again, make sure that there is no residual moisture on the electrical contacts, by drying or blowing on the power plug.
- The electrical components must not be cleaned with a water jet, a high pressure cleaner or other similar device!
Clean only with a moist towel.
- If you suspect that water or any other form of moisture has penetrated the electrical components, pull the power plug out of the wall socket immediately. If already disconnected from the power supply, make sure it is not plugged in again. Clearly mark the nursing bed as "Defective" and take it immediately out of service. Report this to the operator promptly.
- Failure to follow these directions could lead to major damage of the bed and its electrical components and lead to subsequent malfunctions.

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CLEANING AND DISINFECTION PLAN

■ Remove the bed sheets and give them to the laundry.

■ Clean all surfaces including the slatted boards and the reclining surface made of plastic inserts or wire mesh base with a mild and environmentally safe cleaning solution. This also applies to the hand switch.

■ If the bed has visible contamination e.g. infectious or potentially infectious material has polluted it, it must be disinfected by wiping after cleaning. Use approved disinfectants for each surface according to the products listed by the DGHM (German Organization for Hygiene and Microbiology). This also applies to all beds from patients with illnesses subject to report according to §6 of the infection control law (German abbreviation: IfSG, Protection against Infection Act), colonization or infection with treatment resistant pathogens (e.g. MRSA, VRE) and all beds from the intensive care and infection control units. Use only the DGHM approved disinfectant in the approved concentration.

■ Disinfecting the castors is only necessary when visibly contaminated by infectious or potentially infectious material.

Notice: Continuous disinfection is only necessary for patients with a treatment resistant pathogen (e.g. MRSA) within the hospital.

INSTRUCTING THE USER AND QUALIFIED PERSONNEL

To guarantee the correct procedure for cleaning and disinfection is followed, we recommend to correctly instruct the handler and qualified personnel. Thereby communicating that the following points need to be observed:

■ The clean bed is to be transported to the patient's home in such a way that it does not become polluted or contaminated.

■ When disassembling the bed we recommend cleaning and disinfecting it right away. The qualified personnel should be familiar with the steps necessary to thoroughly clean and disinfect (the operator will provide the individual steps necessary). It is necessary to use only the DGHM approved disinfectant in the approved concentration. The disinfectant must be appropriate for the surface being cleaned.

■ The qualified personnel must be equipped with water-tight one-time-use aprons and gloves.

■ Only fresh and clean towels must be used to clean and disinfect, these must then be given to the laundry.

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■ After cleaning the bed, the qualified personnel must also disinfect their hands before going to the next patient. An appropriate disinfectant dispenser should be part of the qualified personnel's gear.

The immediate cleaning of the bed while disassembling has the benefit of not transporting "dirty" beds or bed parts along with clean ones in the same truck. A transfer of potentially infectious spores which may be clinging to the used bed may be avoided. A transfer of spores of a hospital-acquired infection will be avoided with strict adherence to these recommendations.

If the beds are to be stored for a while, they should be covered to keep them free of dust, accidental pollution and contamination.

CLEANING SUPPLIES AND DISINFECTANTS

Make use of these recommendations to keep the bed serviceable for a long time.

■ The surfaces must be intact; any damage should be repaired immediately.

■ We recommend a (damp) wash down. When choosing the cleaning solution make sure it is mild (gentle to skin and surfaces) and environmentally friendly. In general a regular household cleaner can be used.

■ For cleaning the coated metal parts use a wet cloth with regular household cleaners.

■ The disinfectant used for the wash down should be one approved for the purpose and listed by the DGHM (in the recommended dosage). Disinfectants and concentration listed by the Robert-Koch-Institute (the former Bundesgesundheitsamtes) should only be used in case of an epidemic according to the infection safety law and only on the orders of the appropriate health department.

■ Despite the excellent mechanical resistance of the bed, scratches, markings, etc. which penetrate the coating should be resealed using a suitable medium to prevent the penetration of moisture. Contact BURMEIER or your specialized dealer with any questions.

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■ Do not use scouring agents, stainless steel cleaners, abrasive cleaning agents or scouring pads. These substances can damage the surface.

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Notice:

Surface disinfectants based on aldehyde generally have the advantage of working on a large array of pathogens, have a relatively low protein error and are environmentally friendly. Its main disadvantage is the allergy-causing and irritation potential. Compounds on the basis of glucoprotamine do not have this disadvantage and are very effective, though mostly more expensive. Disinfectants on the basis of chlorine splitting combinations, with prolonged contact or in too high doses, can be corrosive to metals, plastics, rubber and other materials. In addition to this, the compounds have a high protein error, are irritants to the mucous membranes and are not environmentally friendly.

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CONTACT WITH DISINFECTANTS

■ Follow the guidelines for each product and manufacturer. Observe the exact dosage! We recommend automatic dosage apparatus.

■ Always mix the solution with cold water, to avoid vapors that aggravate the mucous membranes.

■ Wear gloves to avoid direct skin contact.

■ Do not store the mixed disinfectant in open containers with the cleaning rag inside. Close the container!

■ Use closeable bottles with dosage pumps for wetting the cleaning rag.

■ After finishing the surface disinfection ventilate the room.

■ Use a wipe down disinfecting method, do not use a spray. Using the disinfectant as a spray causes the disinfectant to turn into a mist which could be inhaled.

■ The wipe-down disinfection plays a major part.

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operators of nursing beds are obligated to keep medical products in a safe state throughout their entire service life. This also includes regularly carrying out expert maintenance as well as safety checks.

If a bed is purchased for domestic use (not for commercial use), the regular safety inspection is not compulsory but is still recommended by the manufacturer.

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Notice for Operators

This bed has been constructed and built to last safely for a long time. With appropriate operation and use, this bed has a life expectancy of 2 to 8 years. The life expectancy is based on the conditions and frequency of use.

Through repeated transport, assembly and disassembly, improper operation as well as long time use, it is possible that damage, defects and deterioration can occur. These deficiencies can lead to danger if they are not recognized and quickly fixed.

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Because of this there are legal bases for the regular enforcement of tests to ensure the safe condition of this medical product.

According to § 4 of the Medizinprodukte-Betreiberverordnung operators are obliged to maintain the product. Because of this the following regular inspection and function controls are to be performed by the operator as well as the user.

Point out these necessary tests to the user.

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BY THE USER

Along with the regular substantial tests by technically qualified personnel, the normal user (nurses, supporting relatives, etc.) must also perform a sight and function check at shorter, regular intervals, as well as before a new patient.

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■ If damage or a malfunction is suspected, take the bed out of service immediately and disconnect it from the power until an exchange or repair can be made!

■ Contact your operator regarding exchange or repair of faulty parts.

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Recommendation:

All electrical and mechanical components should be tested monthly. The main power cable as well as the hand switch cable should be checked after every mechanical load and after every relocation.

For this purpose use the checklist on **page 36**.

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Checklist: Test protocol by the user

Test		OK	Not OK	Defect description
Visual check of electrical components				
Hand switch and cable	Damage, Cable Routing			
Power cable	Damage, Cable Routing			
Hand switch	Damage, Foil			
Locking box	Damage, Cable Routing			
Visual check of mechanical components				
Trapeze pole/Socket	Damage, Deformation			
Wooden boards	Damage, Deformation			
Spring wood slats	Damage, Splinters			
Bed box	Damage, Splinters			
Reclining surface frame	Damage, Deformation			
Side rail	Damage, Splinters			
Function test of electrical components				
Hand switch	Functiontest, Blocking function			
Locking box	Functiontest, Blocking function			
Function test of mechanical components				
Emergency lowering of backrest	Test according to instruction manual			
Castors	Locking and unlocking			
Allen screws	Tight fit			
Side rail	Safe locking/unlocking			
Lower leg rest	Latching			
Accessories (e.g. trapeze pole, trapeze)	Secure, Damage			
Signature of the tester:	Result of the test:			Date:

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BY THE OPERATOR

The operator of the nursing bed according to MPBetreibV § 4 is obligated with every new assembly, every maintenance and throughout the operation, to perform regular inspections, to ensure the safe condition of the nursing bed.

These tests are to be repeated during regular maintenance and depending on the use according to MPBetreibV § 4 and from the Directive of the German Employers Liability Insurance Association according to DGUV A3.

- ♦ Observe this sequence for the inspection according to DIN EN 62353:

- I. Visual inspection
- II. Electric Measurements
- III. Function Test

- ♦ The performance of the function test as well as the rating and documentation of the test results can, according to MPBetreibV § 4 only be performed by experts with the appropriate knowledge, training and experience as well as the tools necessary to perform it according to the rules.

- ♦ The performance of electrical measurements according to DIN EN 62353 can, with availability of proper measuring equipment, also be performed by an electro-technically trained person [according to DGUV A3] with medical and appliance specific training.

- ♦ The assessment and documentation of the test results can only be performed by an expert with the corresponding knowledge, training and experience.

Test method:

- Output power test: direct or differential according to DIN EN 62353
- Output power test according to the test unit manufacturers instructions.

Limit:

- Output power less than 0.1mA.

Test cycle:

We recommend a yearly visual inspection and performance check. If this checks was been ok, an electrical testing suffices every three years.

If the 2% error quota (see DGUV A3: §5, table 1 B) is shown to have been observed, the test cycle for inspections and functional testing can be extended to a maximum of two years.

If damage or a function error is suspected take the slatted frame out of service immediately and disconnect it from the power until an exchange or repair can be made!

The test protocol draft on **pages 40 and 41** can be used.

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Customer / med. Facility / practice:						
Address:						
Carried out: <input type="checkbox"/> Repeat inspection			<input type="checkbox"/> Inspection prior initial operation			
<input type="checkbox"/> _____			<input type="checkbox"/> Inspection following repairs / servicing			
Equipment type: <input type="checkbox"/> Hospital bed <input checked="" type="checkbox"/> Nursing bed			Protection classification: <input type="checkbox"/> I <input checked="" type="checkbox"/> II			
Bed type: ROYAL			Inventory Number:			
Location:			Serial Number:			
Grade level (IEC60601-2-52): <input type="checkbox"/> 1 <input type="checkbox"/> 2 <input type="checkbox"/> 3 <input type="checkbox"/> 4 <input type="checkbox"/> 5						
Manufacturer: Burmeier GmbH & Co. KG			User-specific: none			
Testing equipment used (Type / Inventory Number):			1.			
MPG Classification: Class I			2.			
I. Visual Inspection				OK	Not OK	Defect Description
Visual inspection of the electrical components						
Stickers and type plates		Presence, legible				
Housing of transformer unit / Power adapter		Damage				
Motor housing and lifting rods of motors		Correct position, Damage				
Hand switch / Locking box		Damage, Foil intact				
Motor-, hand switch- and power cables		Damage, Routing				
Plug and plug covering strip of control unit		Available, Correct position				
Visual inspection of the mechanical components						
Stickers and type plates		Presence, legible				
Safe working load / patient weight		Presence, legible				
Trapeze pole / socket		Damage, Deformation				
Bed frame		Damage, Deformation				
Spring wood slats		Damage, Splinters				
Castors		Damage				
Reclining surface		Damage, Deformation				
Wooden boards		Damage, Splinters				
Welded seams		Split welded seams				
Side rail		Damage, Splinters				
Knurled screws		Tight fit				
Wearing parts such as joints		Damage				
II. Electric measurements according to DIN EN 62353:2015-10 (leakage current, direct)						
Proceed as follow:				OK	Not OK	Defect Description
1. Plug the power cable into the test socket on the measuring instrument.						
2. Connect the measuring instrument probe with an exposed conductive part of the bed frame (screws, etc.)						
3. Activate the drive motors by using the hand switch for the duration of the measurement.						
4. Start procedures with the measuring instrument						
	Limit Value	Value of first measurement	Current actual value			
Result: Bed SK II (Type B)	0,1 mA	mA	mA			

III. Performance check		OK	Not OK	Defect Description
Performance check of the electrical components				
Motion limit switching of the motors	Automatic cut-off			
Hand switch, blocking functions	Test according to instructions for use			
Control unit and drive motors	Abnormal Noise Development			
Control unit and drive motors	Test according to instructions for use			
Strain relief for power cable	Power cable firmly fastened			
Performance check of the mechanical components				
Joints and pivots	Smooth operation			
Trapeze with belt strap	Secure hold			
Castors	Drive and brake			
Emergency lowering of the backrest	Test according to instructions for use			
Side rail	Safe locking / Secure hold / Safe unlocking			
Lower leg rest	Latching			
Accessories (e. g. trapeze pole, trapeze)	Secure, Damage, Suitability			
Result of the inspection:				
All values within permissible range: <input type="checkbox"/> yes <input type="checkbox"/> no		Next inspection date:		
Inspection passed: <input type="checkbox"/> yes <input type="checkbox"/> no				
If inspection was not passed: <input type="checkbox"/> Defective, do not use the bed! => Repair <input type="checkbox"/> Defective, do not use the bed! => Take out of Service <input type="checkbox"/> Bed does not meet the safety standards				
Test approval sticker applied: <input type="checkbox"/> yes <input type="checkbox"/> no				
Part of this inspection report:				
Comments:				
Inspected on:	Inspector:	Signature:		
Evaluated on:	Operator / Expert:	Signature:		

BURMEIER GmbH & Co KG		
Pivitsheider Straße 270		
D - 32791 LAGE		
☎ + 49 - 05232 - 9841 - 0		
Model	ROYAL	
Type	161741	
Serie	12/04 F	

Model	Bettenmodell
Type	Artikelnummer
Serie	Seriennummer

Model	Bed model
Type	Item Number
Serial	Serial Number

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SPARE PARTS

The corresponding replacement parts can be obtained from BURMEIER by specifying the item number, the order number and the serial number. The necessary details are found on the type plate under the cross tubing of the reclining surface frame.

MANUFACTURER'S ADDRESS

In order to maintain operational safety and the right to claim under warranty, only original BURMEIER replacement parts may be used!

For ordering replacement parts, customer service requests and other questions, please contact:

Burmeier GmbH & Co. KG
(A subsidiary of the Stieglmeyer Group)

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REPLACEMENT OF ELECTRIAL COMPONENTS

Dangerous voltage, electric shock possible

- Before doing any work unplug the main power cable from the wall socket!
- Any work on the electrical equipment may only be performed by customer service, the motor manufacturer or qualified and authorized electricians under consideration of the applicable VDE rules and safety guidelines!

- Only dismantle the drive motors when the reclining surface is in the horizontal position, otherwise there is the risk of being injured by falling parts of the mattress base.
- All the drive components are maintenance-free and must not be opened. In the event of a malfunction, the corresponding components should always be replaced in full!

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Exchange of the Hand switch

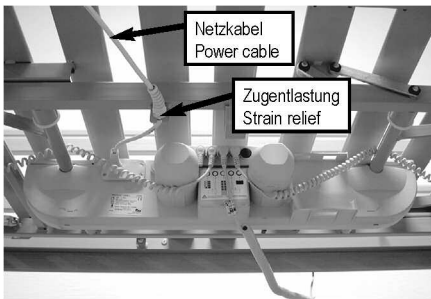
- ☞ Unplug the power plug from the wall socket.
 - ☞ Remove the plug covering strip at the drive unit.
 - ☞ Unplug the plug at the end of the spiral cable of the hand switch from the connector of the drive unit.
 - ☞ Lay the cable of the new hand switch so that it cannot be damaged by moving parts of the bed.
 - ☞ Insert the plug at the end of the spiral cable of the new hand switch into the connector of the drive unit.
- Take care that the O-ring on the plug is not damaged, otherwise the connection may not be moisture-proof.
- ☞ Replace the plug covering strip at the drive unit.
 - ☞ Carry out a function test of the electrical adjustments after the exchange!

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Exchange of the power cable Exchange of the primary fuse

- ☞ Unplug the power plug from the wall socket.
 - ☞ Remove the power cable from the strain relief under the reclining surface frame. You need a wrench SW 19.
 - ☞ Loosen the four screws at the power cable connecting hood.
 - ☞ Pull the power cable connecting hood out of the housing of the drive unit.
 - ☞ Unplug the flat plugs with a flat-nose plier from the fuse sheet bar.
- Now you can replace a defective primary fuse with a similar new fuse.
- A new power cable comes with an integrated power cable connecting hood.
- ☞ Connect the flat plugs of the power cable correctly on the pins of the fuse sheet bar.
 - ☞ Replace the power cable connecting hood orderly, with joint ring, into the drive unit.
 - ☞ Secure the power cable connecting hood with the four screws.
 - ☞ Replace the power cable orderly in the strain relief under the reclining surface frame.
 - ☞ Carry out a function test of the electrical adjustments and a electric measurement test after the exchange!

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TROUBLESHOOTING

The following table on page 46 is a guide to rectifying common malfunctions:

If a function fault is detected during operation, which the user cannot rectify with the use of the trouble-shooting table aid, the specialized staff for maintenance and repair of the appropriate owner is to be called.

The user may **under no circumstances** try to eliminate defects at the electrical equipment!

Dangerous voltage, electric shock possible!

Any work on the electrical equipment may only be performed by service personnel, the motor manufacturer or qualified and authorized electricians as required by the applicable VDE rules and safety guidelines!

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Trouble-shooting table aid

Problem	Possible causes	Solution
Hand switch/central drive unit system not functioning	<ul style="list-style-type: none"> ■ Main power cable is not plugged in ■ Wall socket defective ■ Hand switch plug not properly connected ■ Functions are blocked on hand switch ■ Drives locked at the locking box ■ Hand switch locked (after power failure the hand switch changes to locked status) ■ Hand switch or central drive unit system is defective 	<ul style="list-style-type: none"> ☞ Insert main power cable ☞ Check wall socket/fuse box ☞ Check connector plugs ☞ Unlock functions (see page 23) ☞ Unlock knob at locking box ☞ Change locked status on the hand switch with magnet key ☞ Inform your operator about necessary repairs
Drives only operates for a short time when buttons are pressed	<ul style="list-style-type: none"> ■ Too much weight on the bed ■ Bed is blocked by an obstacle 	<ul style="list-style-type: none"> ☞ Reduce load ☞ Remove obstacle
Function LED on hand switch not lit or lit continuously when button is pressed	<ul style="list-style-type: none"> ■ Defective drive ■ Drive is at end position ■ Locking function is activated ■ Load limit exceeded 	<ul style="list-style-type: none"> ☞ Inform your operator about necessary repairs ☞ Remove drive from end position ☞ Unlock locking box ☞ Reduce load
Individual Drives operate in one direction only	<ul style="list-style-type: none"> ■ Hand switch, drive motor or central drive unit system defective 	<ul style="list-style-type: none"> ☞ Inform your operator about necessary repairs
The wrong function is active when hand switch buttons are pressed	<ul style="list-style-type: none"> ■ Internal motor plugs incorrectly connected (mixed up) 	<ul style="list-style-type: none"> ☞ Inform your operator about necessary repairs
Motor drives stopped suddenly after a long time of adjustments	<ul style="list-style-type: none"> ■ Thermo switch in central drive unit system is triggered on 	<ul style="list-style-type: none"> ☞ Observe minimum pause requirements (see page 21)
Continuous or longer lasting signal without using hand switch	<ul style="list-style-type: none"> ■ Central drive unit system is defective 	<ul style="list-style-type: none"> ☞ Inform your operator about necessary repairs
Mattress base not horizontal	<ul style="list-style-type: none"> ■ Lifting actuators not running synchronously 	<ul style="list-style-type: none"> ☞ Raise or lower mattress base to highest or lowest setting at least once a day.

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Only the Original BURMEIER accessories may be used. If other accessories are used, we can not be liable for any resulting injuries, defects and dangers.

When using side rail, drip/I.V., etc. with electrically adjustable beds, please note:

Make sure that the arrangement of accessories does not produce any crush or shearing zones for the patient when the back and leg rests are adjusted. If this cannot be ensured, the user must safely prevent the patient from adjusting the back- and leg rest.

- To do so, attach the hand switch out of reach (e.g. at the foot end of the bed), or lock hand switch adjustments.

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MEASUREMENTS AND DIMENSIONS
Assembled bed with side rails:

Reclining surface
External dimensions
Total weight
Safe load

Disassembled bed:

Wooden end boards
Chassis head section including motor
Chassis foot section including motor
Reclining surface section with backrest and motors
Four side rail beams
Side boards
Trapeze pole

SPECIAL FEATURES

- ♦ Electrical height adjustment of the reclining surface from about 40 to 80 cm.
- ♦ Electrical adjustment of the backrest from 0° to about 70°.
- ♦ Electrical adjustment of the leg rest from 0° to about 35°.
- ♦ Electrical adjustment of the foot end inclination from about 10°.
- ♦ Simultaneous adjustment of the back and thigh rest (Auto-contour).
- ♦ Rolls on four sikable castors.
- ♦ Reclining surface of 200 x 90 cm, divided into four areas; Outside dimensions are about 220 x 102 cm.
- ♦ Side rails on both sides can be lowered.

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ELECTRICAL DATA

Drive unit

(flange mounted to the backrest motor):

Type
Input Voltage
Max Amps
Output Volts
Output Amps
Operating time
Protection, splash proof
Classification, Protection class II, Type B, MPG classification Class I, not for use in explosive atmospheres.

Power Cable

Type (coiled, anti-kink cable with strain relief, insertable into central drive unit)
--

Hand switch with locking function

Type
Protection

Locking box

Typ
Protection

Motor for reclining surface height adjustment

Type
Power / Size / Lift
Input Voltage
Out Amps
Operating Time
Protection

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Explanation of Icons:

- ♦ Unit with Type B application according to IEC 601-1
(Special protection against electric shock)
- ♦ Protection Class II device, double insulation
- ♦ Transformer with thermal fuse block
- ♦ Safety transformer in accordance with VDE 05510
- ♦ Caution! Consult instruction manual
- ♦ The electrical equipment offers protection from internal dust deposits and is splash-proof from all sides
- ♦ The electrical equipment is dust-proof from internal dust deposits and is splash-proof from all sides
- ♦ Conformity mark in accordance with Medical Products directive 93/42 EWG

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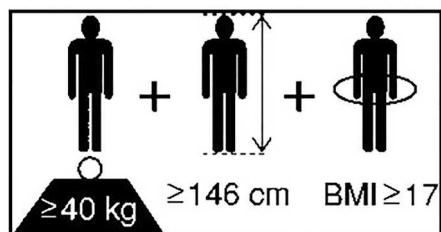
Noise level during motor adjustments

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The ambient conditions stated below must be observed:

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In storage:

Storage temperature
Relative humidity

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In operation:

Ambient temperature
Relative humidity
Air pressure

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Risk analysis for medical devices
Safety for electromedical equipment
Electromagnetic compatibility
Medical electrical equipment: with fitness for use
Medical electrical equipment – Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment
Particular requirements for the safety and essential performance of medical beds
Medical electrical equipment: Particular requirements for the basic safety and essential performance of medical beds

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- ♦ This nursing bed is - if electrically adjustable - classified as a commercially used electric appliance (b2b) in accordance with WEEE-Guidelines 2002/96/EG (Electronics Law).
- ♦ Exchanged electronic components (drives, control unit, hand switch, etc.) of these nursing beds are considered electronic scrap and must be disposed of according to the WEEE-Guidelines.
- ♦ With nursing beds that came into use after August 13, 2005 (see designation "Serial" on the type label at the head of the bed), the operator is required to not dispose of the electrical components in the municipal waste area, but to send them directly to the manufacturer. BURMEIER and their service and sales partners will take these parts back.
- ♦ Our general terms and conditions apply for this redemption.
- ♦ No longer useable rechargeable batteries have to be disposed of according to regulations and do not belong in the house garbage.
- ♦ With all disposed components the owner must make sure that they are not infectious or contaminated.
- ♦ In the event that the nursing bed must be scrapped, the plastic and metal parts used must be disposed of separately, following the proper disposal procedure.
- ♦ Pneumatic springs are under high pressure! Before disposal they have to be depressurized. Please contact the manufacturer of the device on how to depressurize them (see type plate).
- ♦ Hydraulic units are made up of mostly metal and plastic. Before disposing according to manufacturer guidelines the oil has to be drained and disposed of properly.
- ♦ Contact your local community, disposal facility or our service department with any questions.

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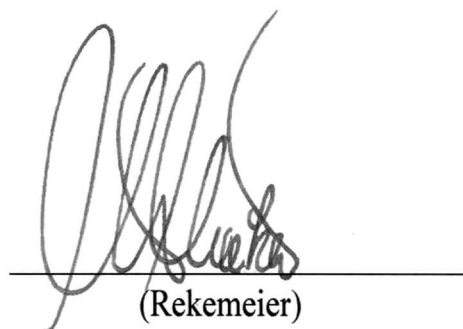
We,

Burmeier GmbH & Co. KG
Pivitsheider Strasse 270
32791 Lage / Lippe
Germany

hereby declare under sole responsibility as the manufacturer that the product model named below:

Nursing bed ROYAL

in the version submitted complies with the regulations of the EC Directive 93/42/EEC for Medical Devices, last amended by Directive 2007/47/EC dated 5 September 2007. It is categorised as a Class I active medical device.

The relevant technical documentation is kept by the manufacturer's safety representative.

To evaluate the conformity to the Directives, all applicable parts of the following standards were referred to:

complies with the regulations of EC Guidelines 93/42/EEG for medical products and fulfills all additional safety requirements by the Highest State Authority from May 22, 2001 for care beds.

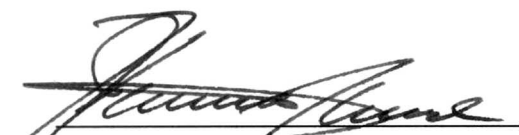
Harmonised standards:

- EN 14971 : 2013-04 Risk analysis for medical devices
- EN 60601-1 : 2007-07 Safety for electromedical equipment
- EN 60601-1-2 : 2007-12 Electromagnetic compatibility
- DIN EN 60601-1-6 : 2010-10 Medical electrical equipment: with fitness for use
- DIN EN 60601-1-11 : 2010-10 Medical electrical equipment – Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment
- DIN EN 60601-2-52 : 2010-12 Particular requirements for the safety and essential performance of medical beds


International standards:

- IEC 60601-1-2-52 : 2009-12 Medical electrical equipment: Particular requirements for the basic safety and essential performance of medical beds

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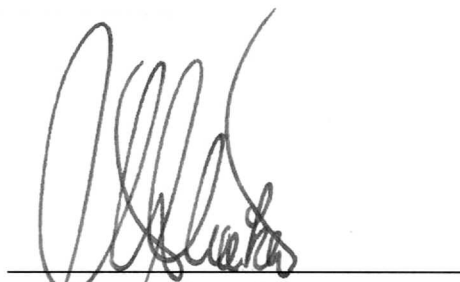No part of this manual may be reproduced with-
out the prior written permission of the publisher!

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All rights reserved.
Subject to technical changes!

Stand: 2017.06.28

Updated: 2017.06.28