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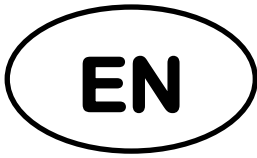
Safety side cushions

Assembly and operating instructions

For use on various Burmeier nursing beds with safety sides

2022-11-23 | Version 1 | 300595

Use for the intended purpose

- For use on various Burmeier care beds for cushioning and reducing the gap between safety sides, according to the specifications of these operating instructions and as required by the clinical condition of the resident.

Intended use

- To protect particularly restless, confused residents/patients from injury should they bump against raised safety sides.
- To further reduce the safety side gaps and thus provide additional protection for particularly restless, confused or very emaciated, slender residents/patients against the possible risk of entrapment or strangulation when the safety sides are raised.

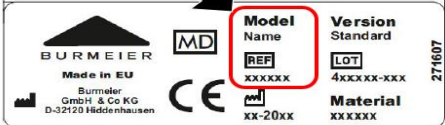
Product description

- Textile safety side cushion for covering safety sides on Stieglmeyer hospital and care beds.
- The inside of the cushion is made of steam-treated rayon/interlock fabric.
- Outer cover made of foam leather, with additional foam and inner lining on the inner side.
- Zipper or snap fasteners for fastening around the safety sides.
- The outer material of the cushions is made of hard-wearing, high-quality "SKAI" faux leather.

Assembly instructions

How to identify existing beds, safety sides and safety side cushions

Depending on the bed model, there are one or more types of safety sides that may be permanently fitted to the bed or can be fitted separately as accessories. To identify them, please look at the type plates, instructions for use and the information in these instructions. If the version you have is not described here, please get in touch with your Stieglmeyer medical product advisor or our customer service (see →page 17 for contact details).

<p>Bed model</p>	<p>Type plate on the frame of the mattress base, on the inside at the head end</p>	 <p>The image shows a type plate with the Burmeier logo, 'Made in EU', and 'CE' marks. It contains fields for 'Model Name' (with 'REF' and 'xxxxxx' below it), 'Version Standard' (with 'LOT' and '4xxxx-xxx' below it), and 'Material' (with 'xxxxx' below it). A red box highlights the 'Model Name' field, and a black arrow points to it from above.</p>
<p>Type of safety side</p>	<p>The description in these instructions, in the bed's operating instructions, or, in the case of accessory safety sides, on their type plate</p>	 <p>The image shows a type plate with the Burmeier logo, 'Made in EU', and 'CE' marks. It contains fields for 'REF' (with 'xxxxxx' below it) and 'LOT' (with 'xxxxxxxxxx' below it). A red box highlights the 'REF' field, and a black arrow points to it from above.</p>
<p>Type of safety side cushion</p>	<p>The description in these operating instructions, REF or on the type plate in the inner lining of the safety side cushions</p>	 <p>This image is identical to the one in the previous row, showing a type plate with 'REF' and 'LOT' fields highlighted in red boxes with arrows pointing to them.</p>

The versions of safety side cushions available are continually developed and expanded. For this reason, it is possible that you will not find an example **REF** in this instruction manual that corresponds to the particular variant indicated on your type plate. If this is the case, please refer to the detailed illustrations and descriptions on the following pages to help you identify and fit your particular type of safety side cushion.

Installing the safety side cushions

WARNING

Risk of injury

Failure to observe these warnings may result in an increased risk of injury to the resident due to entrapment between raised safety sides or due to malfunctions in the safety side locking system.

- Only use undamaged protective covers and safety sides.
- Only use protective covers that match your existing safety sides and always attach the covers properly in accordance with these instructions for use. If you have any questions or concerns, please contact Burmeier's customer service department.
- After installing the protective cover, check that the safety sides still perform their protective function; in particular make sure that the safety sides still engage properly in the raised position.
- Observe the safety instructions for the safety sides in the corresponding operating instructions for the beds.

Note on illustrations

Please note: The illustrations of the cushions/safety sides used in these assembly instructions may differ from the actual appearance of your particular cushions/safety sides.

The deviations in the illustrations have no influence on the assembly or functionality, however.

Attaching the safety side cushions

The assembly procedure is identical for the right and left safety sides
To remove the cushions, follow the instructions in reverse.

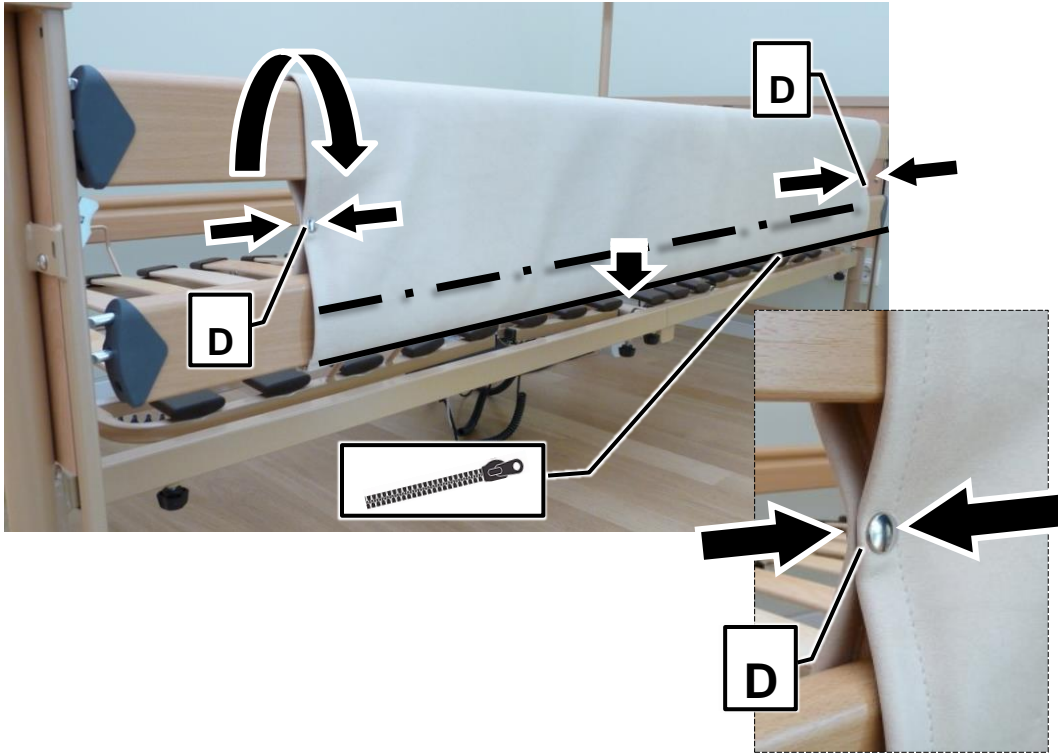
Wrap-around tube

Bed model	Type of safety side	Type of safety side cushion	REF
Allura Arminia Dali Fortuna Gigant Inovia Regia Westfalia	Full-length safety sides (DSG) 	 For full length safety sides, 2 bars; + extension bar: Fastened with zip/snap fasteners	For 2 bars: 301725 +extension bar: 301722
Gigant Lenus	Combinable safety sides (KSG) 	 Fastened with zip/snap fasteners	Head end (short): 301728
Regia	TSG - Vario Safe – split safety side  Can be fitted to the bed before delivery, or be removable with quick-release fastener, or be fitted later as an accessory 	 Length: 90 cm + 110 cm Fastened with snap fasteners	Length 90cm: 301732 Length 110cm: 301731

Wrap-around tube for Full-length safety sides (DSG)

<p style="text-align: center;">1</p> 	<p style="text-align: center;">2</p> 
<p>Lay the safety side cushion over the safety side – with the faux leather side on the outside!</p>	<p>Fit the two zip ends together.</p>
<p style="text-align: center;">3</p> 	
<p>Zip up the zip fastener completely.</p>	

4

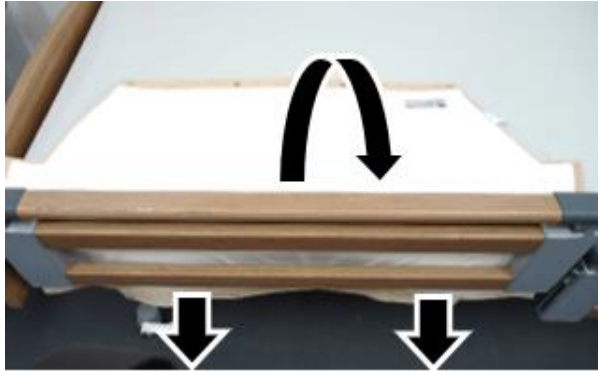
Slide the safety side cushion around until the zip is right at the bottom and the snap fasteners [D] at the sides are aligned with each other.

Fasten the snap fasteners [D] (at the left and right sides of the cushion) – done!

i If a top rail is mounted on the safety side, a safety side cushion with a larger height dimension must be fitted. Please contact our customer centre for this.

The safety side can be operated normally after fitting the safety side cushion.

Wrap-around tube for TSG - Vario Safe – split safety side

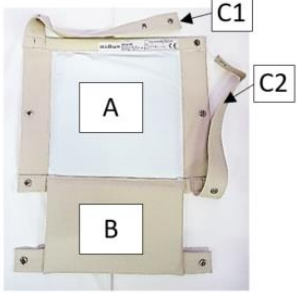
1	2
	
<p>Rise safety sides (to 1. or 2. Step; shown: 1. step) Place the safety side cushion as shown on the mattress Lining fabric upwards; straight edge points to the split safety sides</p>	<p>Pull half of the safety side cushion down into the gap between the split safety side and the mattress. Place the remaining part (with slanted edge) over the split safety side at the top.</p>
3	Alternative: 1a; 2a; 3a
	
<p>Under the lower safety side bar: Close the four press studs by hand. Align the safety side cushion in the centre - Done</p>	<p>Split safety side is fully raised (2nd step): Carry out steps 1 - 3 in the same way, but pass the safety side cushion around under the 2nd bar (from below) and close it.</p>

 **Attention - Please do not do that!**

	<p>Note the safety side cushion on the inside/outside!</p> <p>The bevelled part of the safety side cushion must always be visible on the outside of the bed for maximum patient protection.</p>
	<p>Note the lengths of the safety side cushions!</p> <p>Safety side cushion lengths 90/110 cm must match corresponding safety side lengths for max. patient protection - do not interchange!</p>

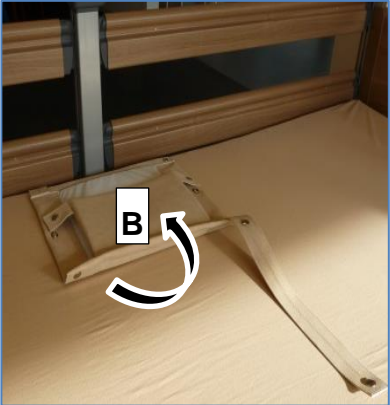
Protective cushion for KSG mobilisation support

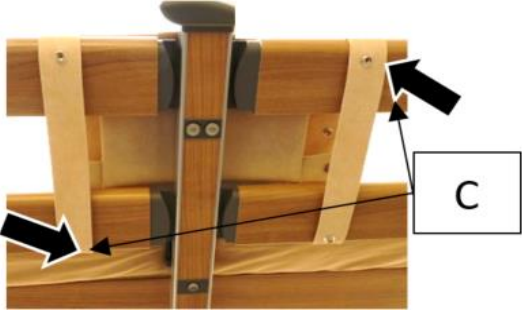
Bed model	Type of safety side	Type of safety side cushion	REF
Arena Basano; Elvido;	Combinable safety sides (KSG) 	Protective cushion for mobilisation support 	With top section (to cover the knob on the mobilisation support): 712961 Without top section: 703186

As delivered:		<p>A: Cushioned section with snap fasteners</p> <p>B: Top section with snap fasteners</p> <p>C1; C2: Fastening straps with snap fasteners</p>
---------------	--	--

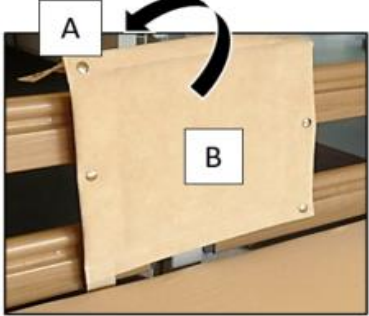
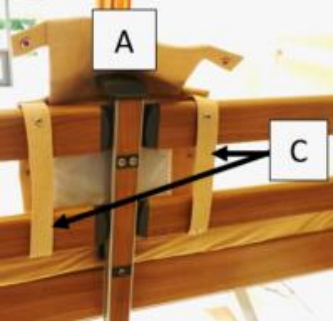
There are 3 possible installation options!

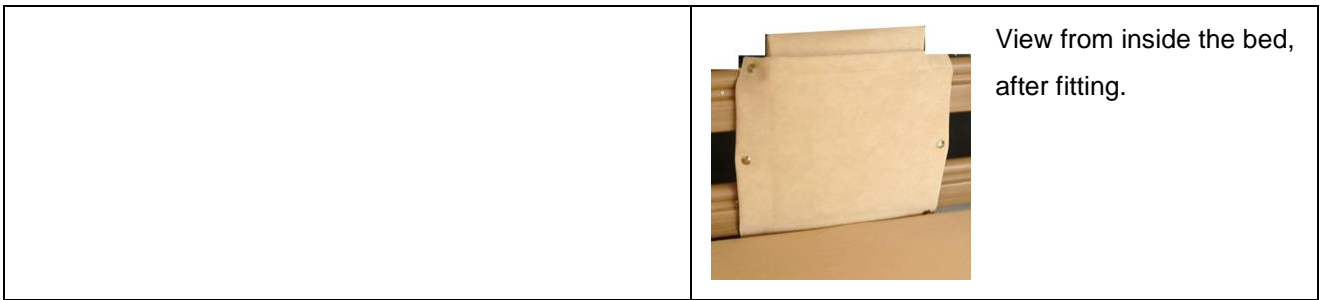
Version 1: Safety side raised on both sides (button on mobilisation support remains uncovered)

1	2
	
<p>Starting position, with the cushion laid on the mattress (view from inside the bed):</p> <p>Fold the top section [B] over onto the cushioned section [A]</p>	<p>Lay the cushion sections [A+B] centrally on the inner side of the bed, in front of the mobilisation support, and fold them upwards</p> <p>Lay both the fastening straps over the safety side bars, towards the outside of the bed</p>

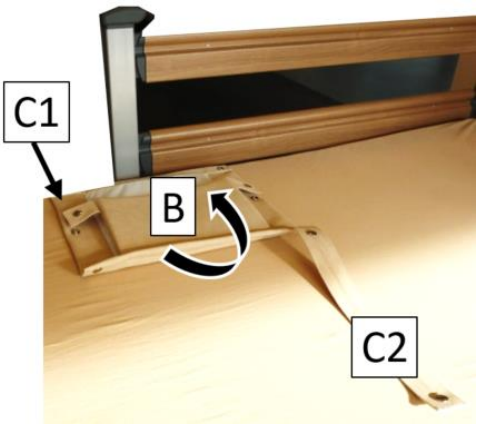
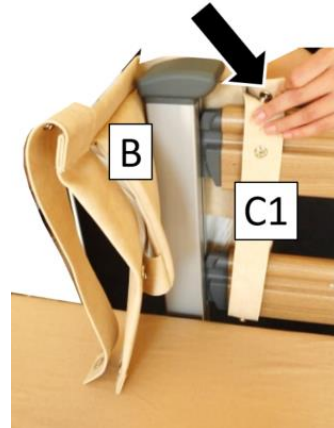
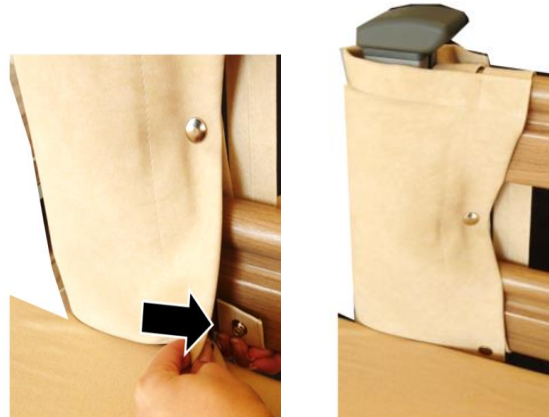
3	
	
<p>(View from outside the bed) Attach both fastening straps [C] to the cushion section [A+B] with the snap fasteners. Done!</p>	

Version 2: Safety side raised on both sides (button on mobilisation support is covered)

1	2
	
<p>Starting position, with the cushion laid on the mattress (view from inside the bed): Top section [B] remains folded out.</p>	<p>Lay the cushion [A+B] centrally over the mobilisation support. Fold cushion section [A] over the knob and towards the outside of the bed; cushion section [B] remains on the inner side of the bed.</p>
3	4
	
<p>(View from outside the bed) Attach both fastening straps [C] to the cushion section [B] with the snap fasteners.</p>	<p>Fasten the top section [A] to the fastening straps using the two snap fasteners. Done!</p>

Version 3: Safety side raised on one side (mobilisation support is covered)

<p style="text-align: center;">1</p> 	<p style="text-align: center;">2</p> 
<p>Starting position, with the cushion laid on the mattress (view from inside the bed): Fold the top section [B] over onto the cushioned section [A]</p>	<p>Place the cushion around the outside of the support post. Wrap the fastening strap [C1] around the bars and fix at the top with the snap fastener.</p>
<p style="text-align: center;">3</p> 	<p style="text-align: center;">4</p> 
<p>Wrap the cushion around the post and fasten the middle snap fastener.</p>	<p>Fasten the bottom snap fastener. Take the fastening strap [C2] and tuck it away between the cushion layers. Done!</p>

Cleaning and disinfection

Clean and disinfect the cushions before using them for the first time, before each reuse and regularly, as needed.

Cleaning

The fabrics used are particularly resistant to dirt and therefore require only minimal cleaning.

- Do not dry clean or chemically clean this safety side cushion

Should soiling occur, however, please proceed in accordance with the following manufacturer's instructions for cleaning the "SKAI ® Palena" faux leather material: (See also: https://www.skai.com/fileadmin/user_upload/Interior/Downloads/Reinigung-Pflege/Reinigung_und_Pflege.pdf)

- Use a warm, mild soapy solution and a microfibre cloth. For heavier soiling, a soft hand brush can also be used.
- Soiling caused by oil, grease, foods with strong colouring, or other substances must be removed immediately. If there is heavy soiling, the cleaning process may have to be repeated. When using commercially available surfactant cleaning agents, please always follow the dosage instructions of the manufacturer.
- Regular cleaning of your product is sufficient to maintain its service life. Additional treatment, using leather care products, for example, is not necessary.
- Please do not use chemical bleaches or abrasive cleaning agents.
- Please note that staining of the material caused by jeans or other textiles is not covered by any warranty.

Disinfection

Cleaning should always be followed by disinfection.

- For further details on recommended disinfectants, please refer to the website of the manufacturer of the "SKAI ® Palena" faux leather used: https://www.skai.com/fileadmin/user_upload/Interior/Downloads/Reinigung-Pflege/skai_Desinfektionsmittelempfehlung.pdf
- After wiping with a disinfectant, always finish by wiping the faux leather with a dry cloth.

Maintenance

The safety side cushions are maintenance-free.

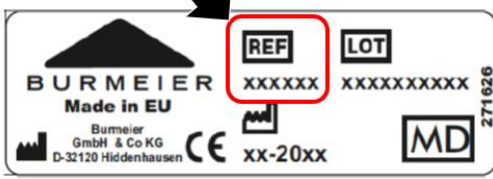
Manufacturer/Customer service

In Germany, please contact our customer service department for spare parts orders, customer service requests and any other queries:

Burmeier GmbH & Co. KG
(A company of the Stieglmeyer Group)
Industriestraße 53 • D - 32120 Hiddenhausen
Tel. :+49 (0) 5223 9769 - 0
Fax:+49 (0) 5223 9769 - 090
E-Mail: info@burmeier.com
Internet: www.burmeier.com

Customers outside Germany can contact our distribution companies in their particular country if they have any questions. Contact details can be found on our website.

You will find the necessary details on the type plate on the inner lining/inner side of the cushion.

	Meaning of the symbols	
	 Article number	 Production batch
	 Production date	 Conforms to Regulation (EU) 2017/745 (MDR)
	 Manufacturer	 Medical device

Technical data

Outer cover/upper material:	Material thickness: approx. 1.25mm; robust, high-quality “SKAI-Palena” faux leather, suitable for use in a medical setting; bio-compatibility tested; 90% polyvinyl chloride, 10% polyester/cotton knitted fabric
Outer cover/upper material meets the fire protection norms:	NF-P 503 Class M2, BS 5852 IS 0+1, low flammability according to EN 1021-1+2, DIN 53438-2+3, DIN 4102-1 B2, MVSS 302, ISO 3795, DIN 75200, Directive 95/28/EC, ÖNORM B 3825
Inside – cushioning:	Steam-treated rayon/interlock fabric/foam
Inside – lining material:	PU covering fabric
Fasteners:	Zip/snap fasteners

Classification

Conforms to Medical Device Regulation (EU) 2017/745 (MDR)		UMDNS code:	17-188 (cushion, bed grille)
Class I medical product		EMDN code:	V08068099 (MEDICAL BEDS - OTHER ACCESSORIES)

Disposal instructions

The safety side cushion contains synthetic and textile components. It must therefore be disposed of properly as residual waste, in accordance with the local regulations.

Declaration of conformity

We, Burmeier GmbH & CO. KG, hereby declare, under sole responsibility as the manufacturer, that the product model named below: “safety side cushion” in the version submitted complies with the REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND THE COUNCIL of 5 April 2017 (MDR). It is categorised as a Class I medical product.

The applicable complete current declaration of conformity is available on request from our customer centre and on our website in the dealer area.

Market information

This product is not licenced for use on the North American market. This applies particularly to the United States of America. The distribution and use of the care bed in these markets, including through third parties, is prohibited by the manufacturer

Burmeier GmbH & Co. KG

(A company of the Stiegemeyer Group)

Industriestraße 53 • D - 32120 Hiddenhausen

Tel. :+49 (0) 5223 9769 - 0

Fax:+49 (0) 5223 9769 - 090

E-Mail: info@burmeier.com

Internet: www.burmeier.com

No part of this manual may be reproduced without the prior written permission of the publisher.

All rights reserved.

Subject to technical changes without notice!

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