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
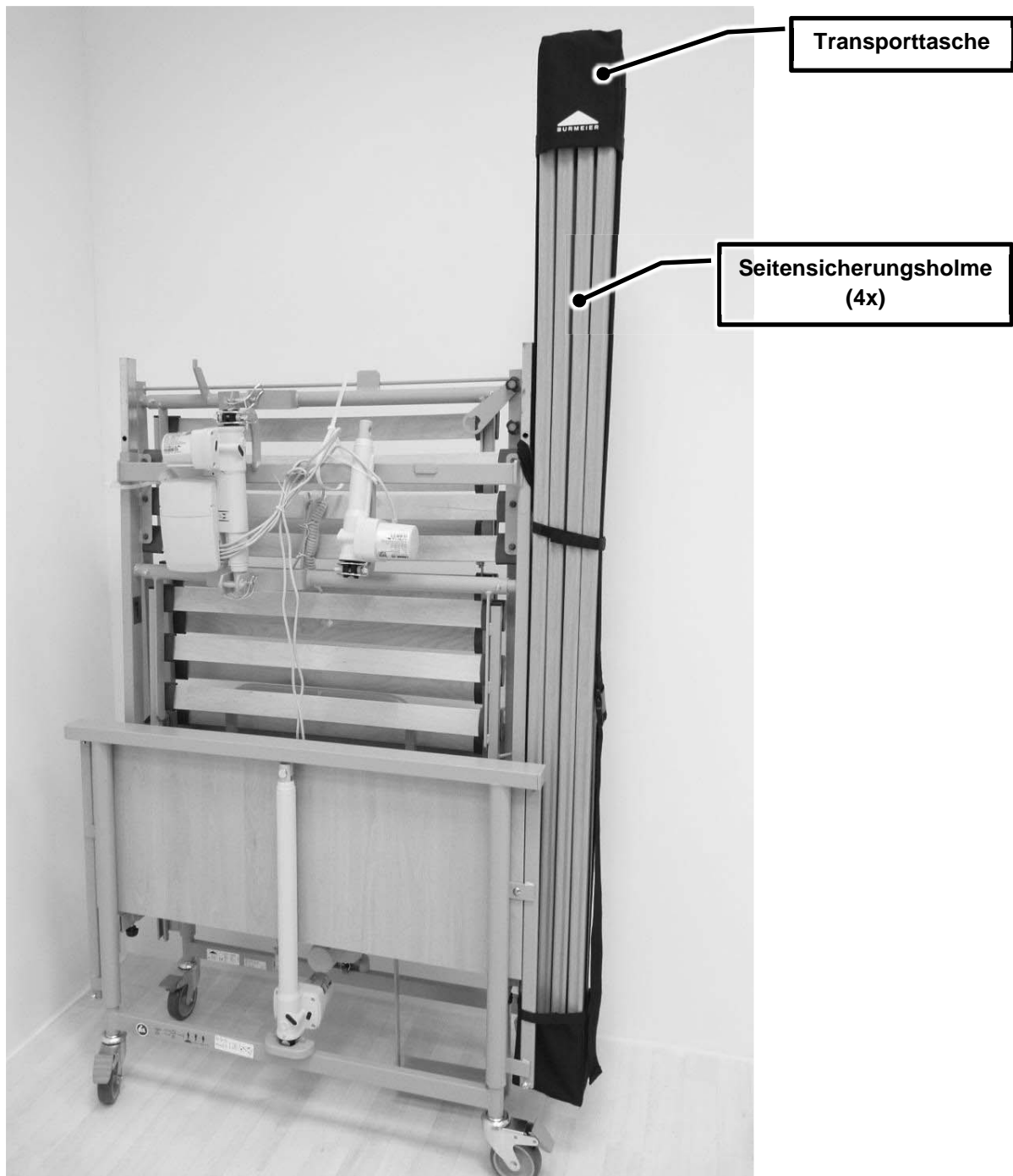
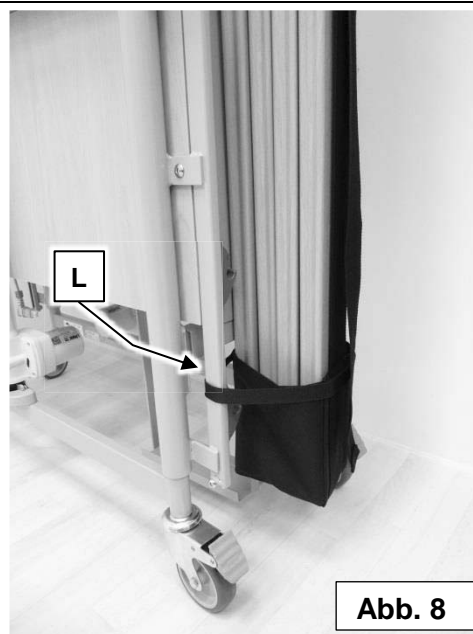
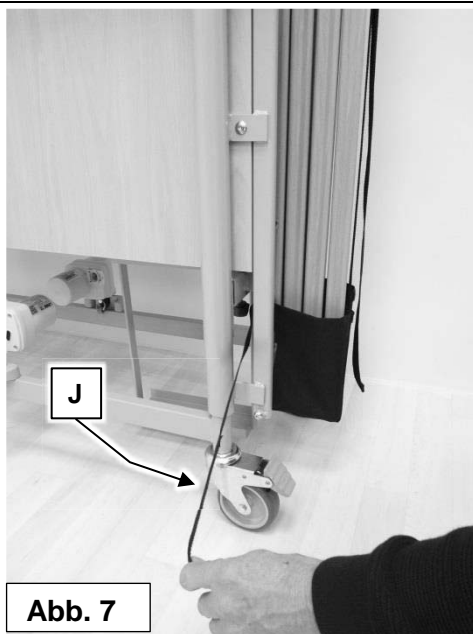
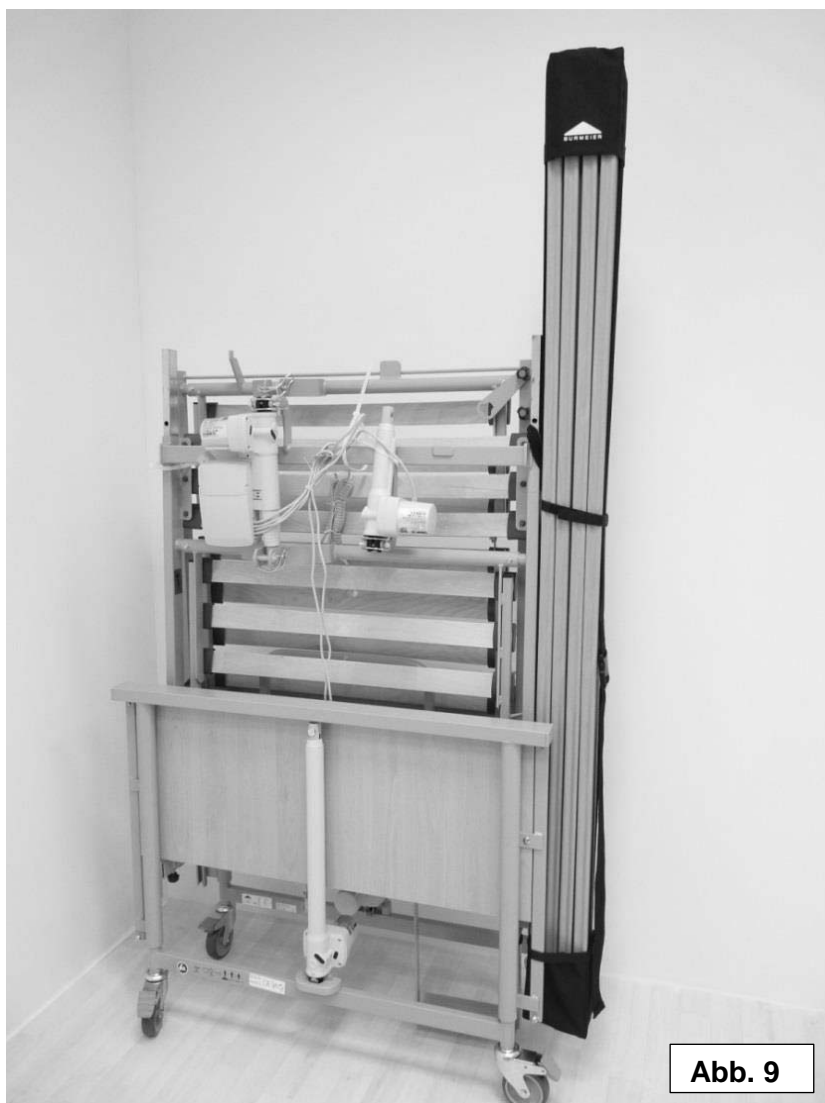
29.01.2019 / V1 / 277562





Mounting instructions for the safety side bar transport bag

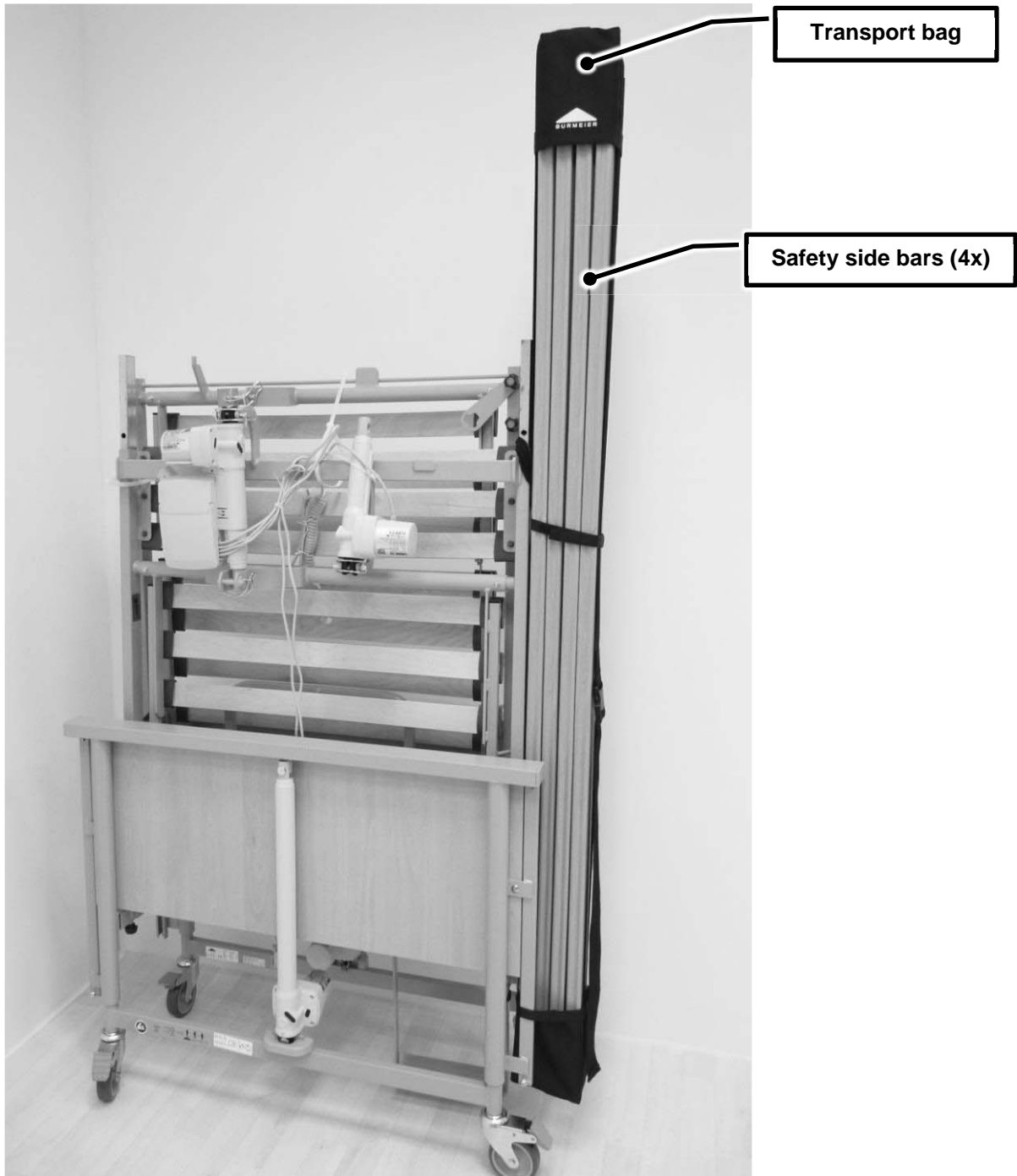In the transport bag can be stored the full-length safety side bars (4 bars) for Burmeier care beds. It can be attached securely to the storage aid and transported as described in the following pages.

Continue on the next page

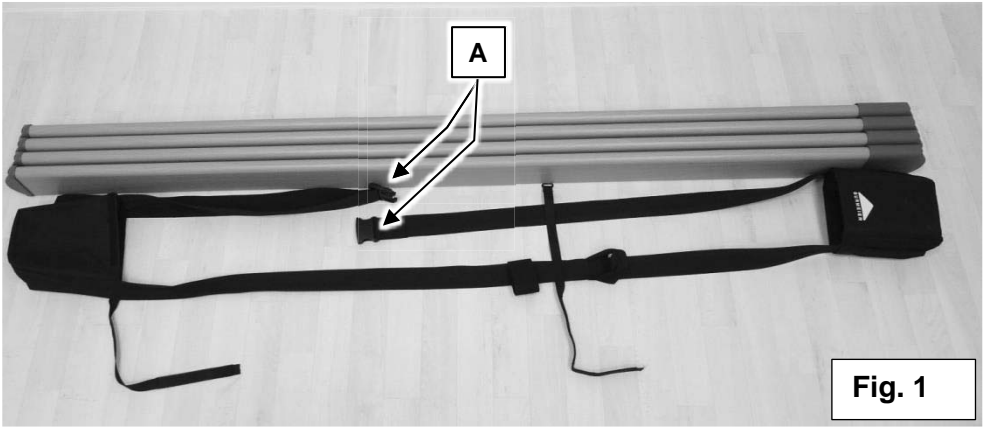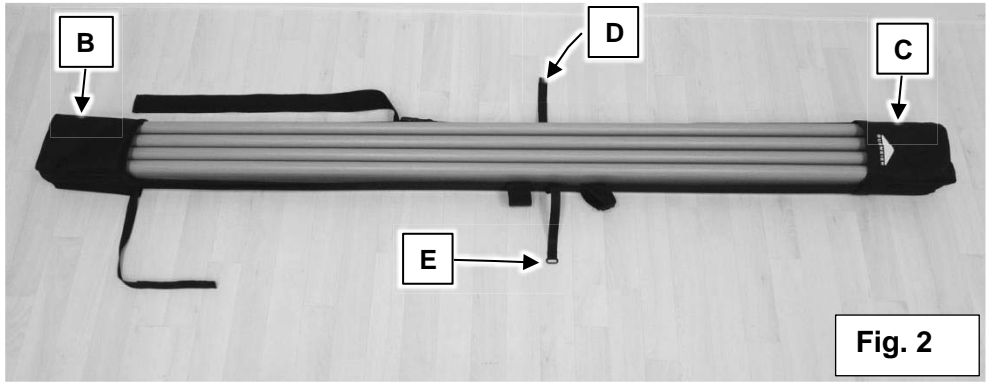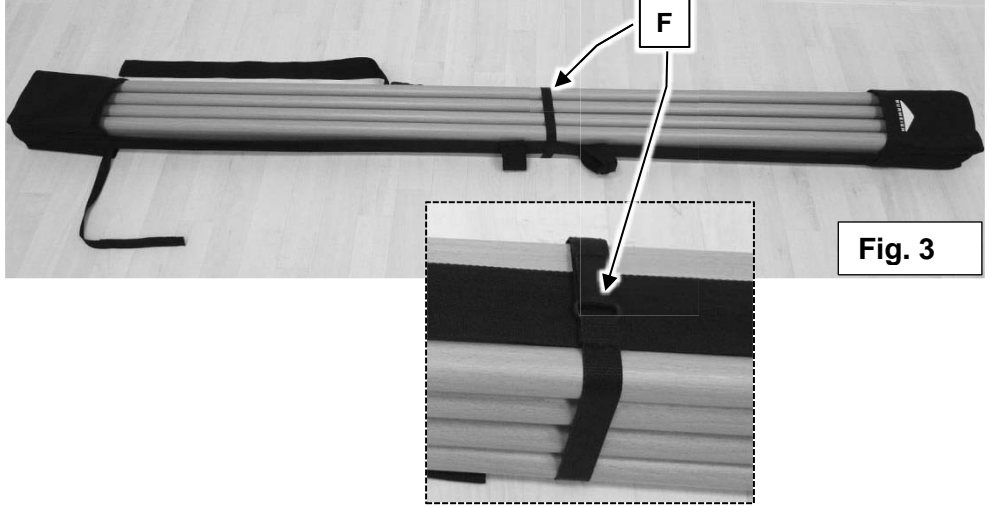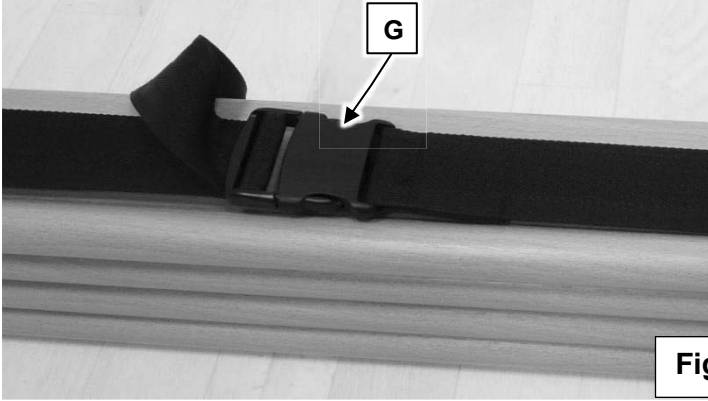Proceed as follows:


<p>1. Place the 4 bars on the floor as shown.</p> <p>Place the transport bag directly next to the bars.</p> <p>Note: The Burmeier logo is facing up.</p>	 <p>Fig. 1</p>
<p>2. Insert the bars in the pockets B and C as shown.</p> <p>Pull the velcro tape D under the bars.</p>	 <p>Fig. 2</p>
<p>3. Insert the velcro tape D into the ring E (Fig. 2), pull it tightly and close the Velcro fastener F.</p>	 <p>Fig. 3</p>
<p>4. Close the strap's side-release buckle A (Fig.1) G (Fig. 4).</p> <p>Adjust the strap as required (lengthen/shorten).</p> <p>Note: Pull the strap tight and flat before closing. The strap must not be twisted/knotted.</p>	 <p>Fig. 4</p>

5. Place the bars with the transport bag upright next to the bed.


Please note the following:

The carrying loops **H** and **I** are used for hanging the bag to the storage aid. Depending on the bed model, use one or the other loop.

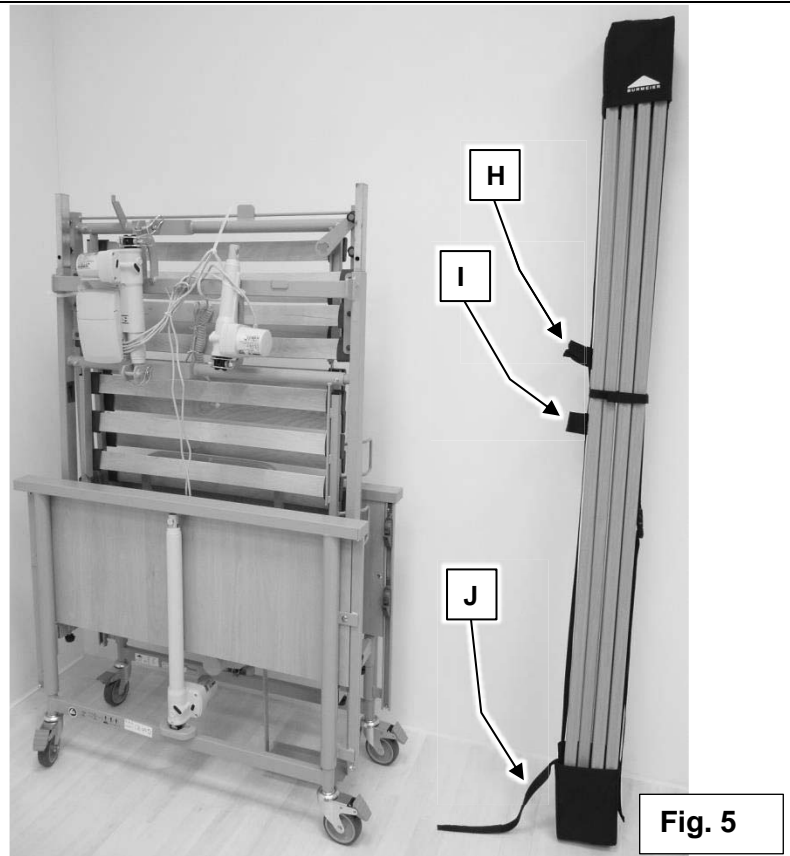The velcro tape **J** is used for attaching the bars at the bottom of the storage aid.

6. Attach the bars using the carrying loop **H** or **I** (depending on the bed model) to the frame of the mattress base **K** as shown (see Fig. 6.1).

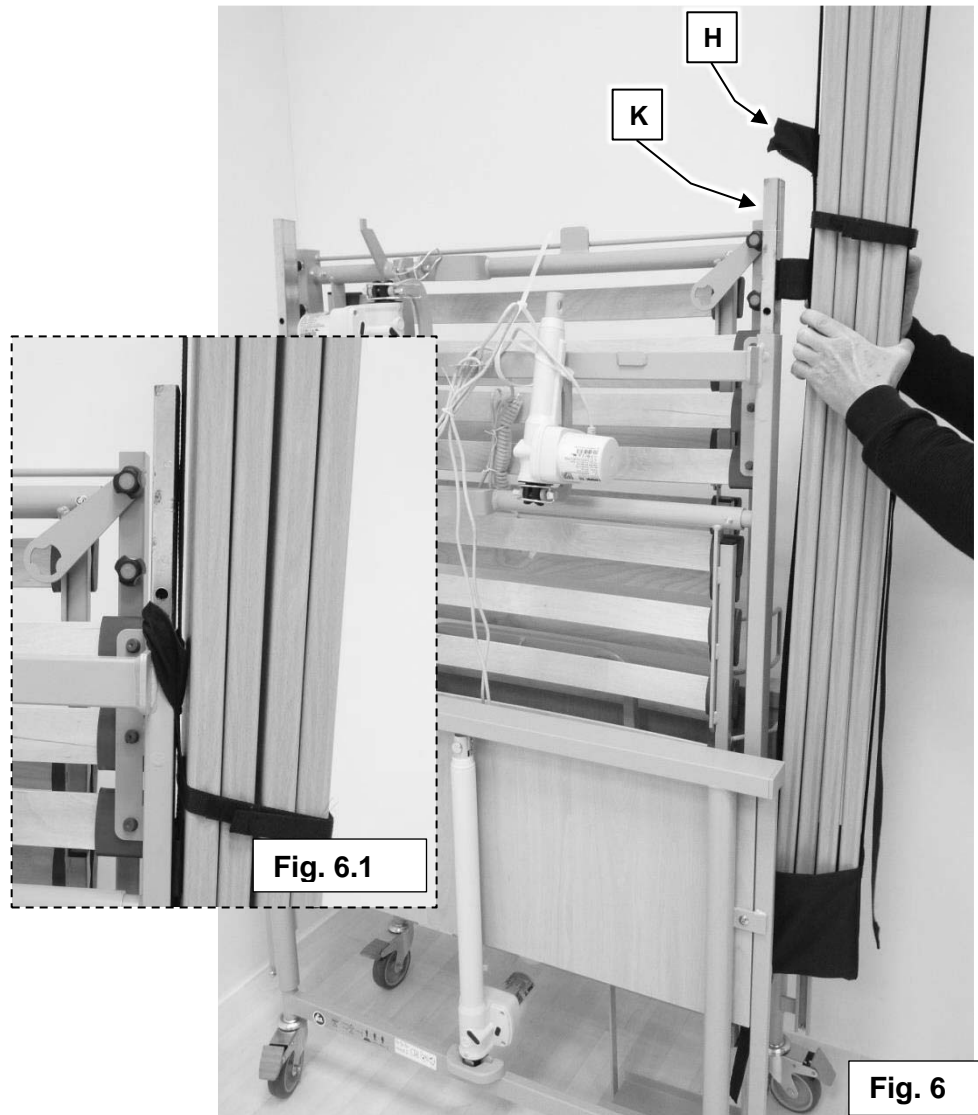7. Attach the transport bag securely to the storage aid **L** using the velcro tape **J** as shown.

Note:

Before transport, make sure that the transport bag is securely attached to the storage aid.

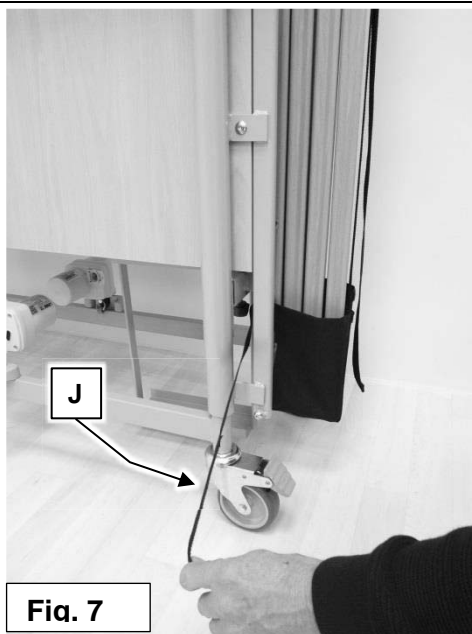
Fig. 7

Fig. 8

The mounting procedure is now completed.

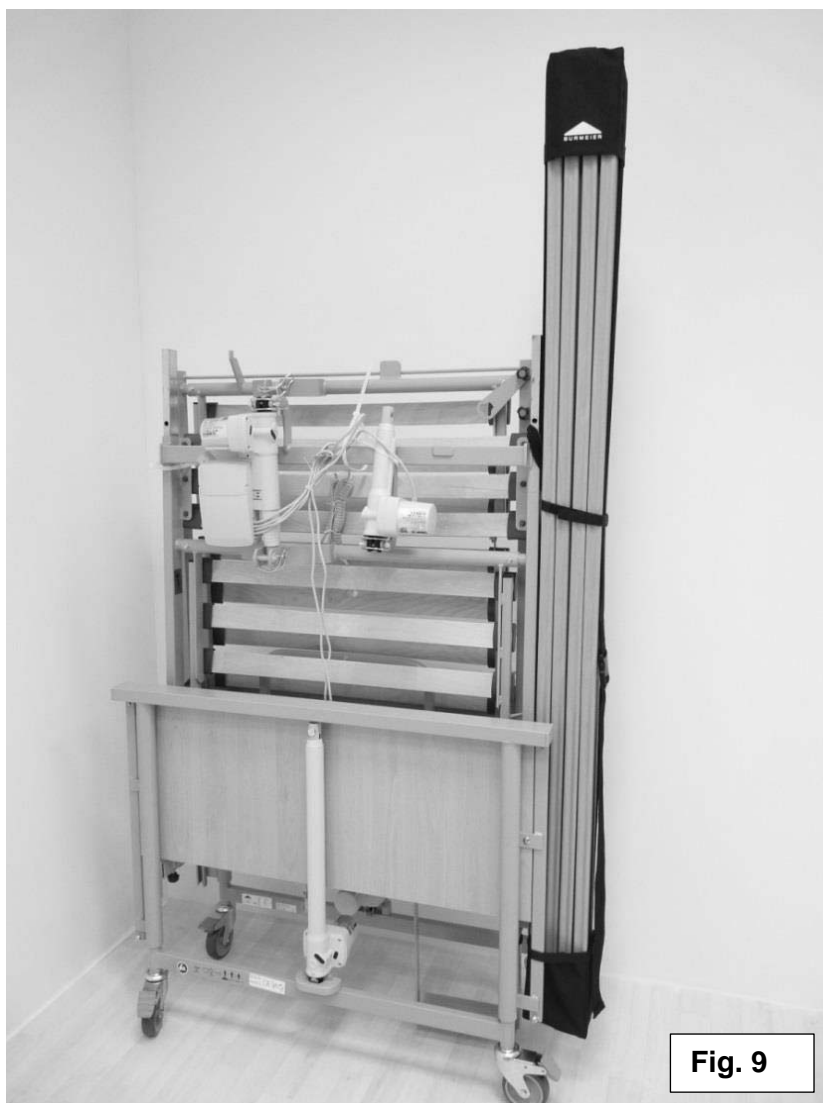
Fig. 9

Burmeier GmbH & Co. KG
Pivitsheider Straße 270, 32791 Lage/Lippe
Telephone: + 49 (0) 5232 / 9841- 0
Fax: + 49 (0) 5232 / 9841- 4
www.burmeier.de
info@burmeier.com

29/01/2019 / V1 / 277562