
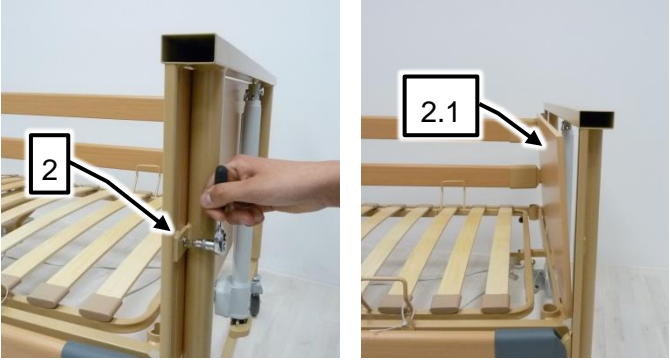
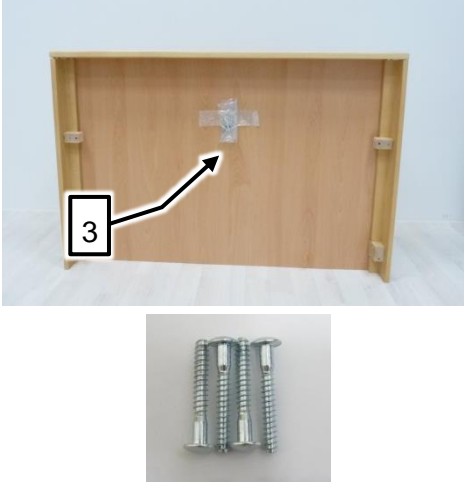
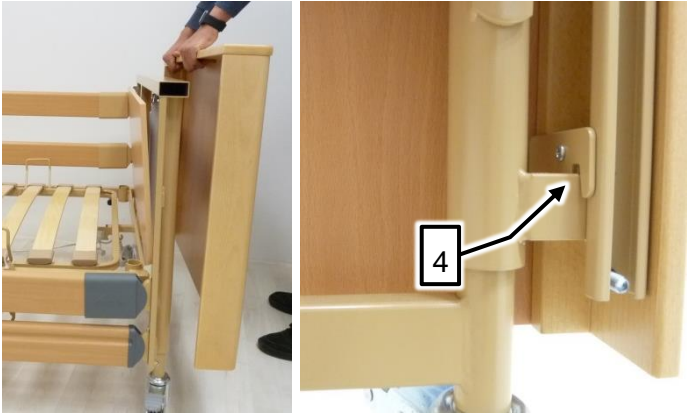


## Montageanleitung der Holzhaube für das kopf- bzw. fußseitige Fahrgestell

Gehen Sie wie folgt vor:

<p>1.</p>	<p>Bett für die Montage vorbereiten: Entfernen Sie die beidseitig eingesteckten Stopfen <b>[1]</b> aus dem Querprofil des Fahrgestells.</p>	
<p>2.</p>	<p>Demontieren Sie auf der linken und auf der rechten Seite die beiden Schrauben <b>[2]</b>, sodass das Füllbrett gelöst ist <b>[2.1]</b>.</p>	
<p>3.</p>	<p>Entnehmen Sie nun die neuen Schrauben aus der Tüte, die sich auf der Rückseite der Haube befindet <b>[3]</b>.</p>	
<p>4.</p>	<p>Setzen Sie als nächstes die Haube so auf das Fahrgestell, dass die Haken auf der Innenseite der Haube (auf beiden Seiten) genau in die Laschen des Fahrgestells greifen <b>[4]</b>.</p>	

5.	Fügen Sie nun das Füllbrett des Fahrgestells und die Haube mit den neuen Schrauben (aus Schritt 3) zusammen <span style="border: 1px solid black; padding: 0 2px;">5</span> .	
6.	Endergebnis der Montage	

**Burmeier GmbH & Co. KG** (Ein Unternehmen der Stieglmeyer-Gruppe)

Pivitsheider Straße 270, 32791 Lage/Lippe - Telefon: + 49 (0) 5232 / 9841- 0 - Fax: + 49 (0) 5232 / 9841- 41

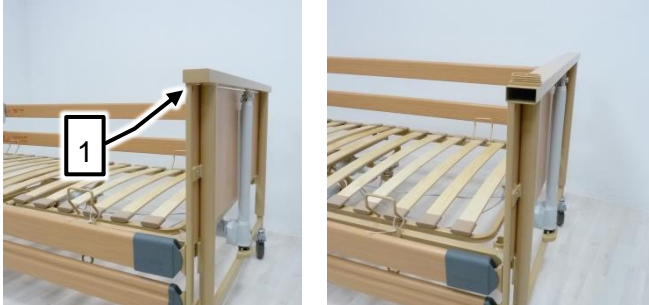
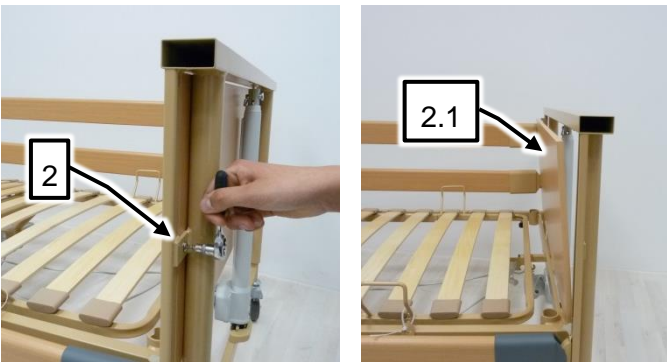

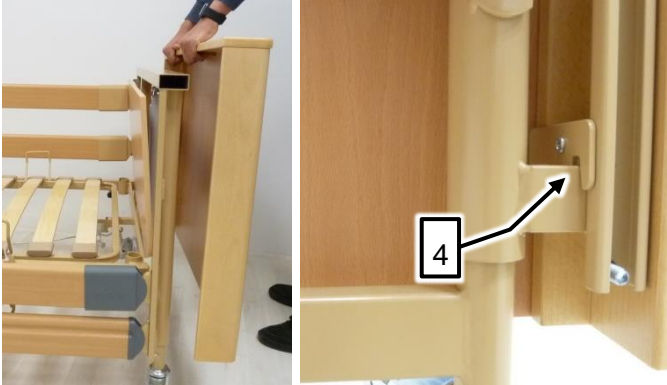
[www.burmeier.de](http://www.burmeier.de) E-Mail: [info@burmeier.de](mailto:info@burmeier.de)

2019-04-18 | Version 0 | 279525

# Dali care bed EN + DE

## Installation instructions for attaching wooden surround to chassis head or foot end

Proceed as follows:

<p>1. Prepare the bed for installation: Remove the plastic stoppers <b>1</b> from both ends of the chassis cross profile.</p>	
<p>2. Remove the two screws on the left and right-hand side <b>2</b> to release the panel <b>2.1</b>.</p>	
<p>3. Take the new screws out of the bag on the back of the wooden surround <b>3</b>.</p>	
<p>4. Fit the wooden surround to the chassis so the hooks inside the surround (on both sides) fit exactly over the lugs on the chassis <b>4</b>.</p>	

5.	<p>Now assemble the chassis panel and the wooden surround using the new screws <b>5</b> (refer to step 3).</p>	
6.	<p>Final result showing wooden surround fitted</p>	

**Burmeier GmbH & Co. KG** (A Stieglmeyer Group company)

Pivitsheider Straße 270, 32791 Lage/Lippe, Germany - Phone: + 49 (0) 5232 / 9841 - 0 • Fax: + 49 (0) 5232 / 9841- 41

[www.burmeier.de](http://www.burmeier.de) Email: [info@burmeier.de](mailto:info@burmeier.de)

18/04/2019 | Version 0 | 279525